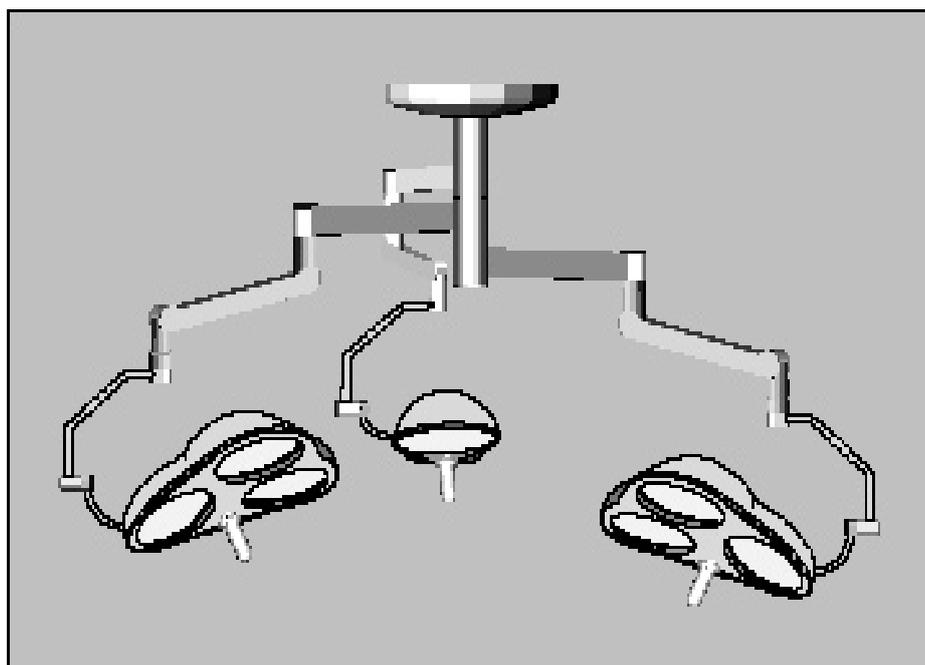
HANAULUX®
blue 130/100/90

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Operating instructions

56 351 001/D

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Seite

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Contents**Page**

1	Instructions for safe operation	4
1.1	Explanation of symbols	5
1.2	General safety instructions.....	6
2	Operating the HANAULUX® <i>blue 130/100/90</i> Luminaire System.....	8
2.1	Components of the Luminaire system.....	8
2.2	Operating the lamp.....□	8
2.3	Function of the Light System.....	8
3	Cleaning and disinfection.....	10
3.1	Cleaning/Disinfection .□	10
3.2	Sterilizing the handles □	10
4	Testing and servicing....□	11
4.1	Testing.....□	11
4.2	Servicing	11
5	Maintenance □	12
5.1	Changing the lamp.....□	12
5.1.1	Changing the lamp <i>blue 130</i>	12
5.1.2	Changing the lamp <i>blue 100</i>	14
5.1.3	Changing the lamp <i>blue 90</i>	14
5.2	Adjusting the spring force.....	16
5.4	Adjusting the brake force	18
5.5	Consumable material .□	18
6	Fault finding.....□	20
7	Technical data.....□	22

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8/2000

Dear user:

- These instructions for use are intended for medical and technical staff in the hospital in the doctor's surgery.
- The work in the „Maintenance“ section may be carried out by one of the operator's service technicians, observing the safety instructions,
- The cleaning of the apparatus may be done by instructed cleaning staff.

These operating instructions apply to all HANAULUX® *blue 90/ 130* ceiling models.

- Please read these operating instructions carefully before using the apparatus. You will then enjoy all the advantages the apparatus offers and protect yourself and others from harm.

Dear Operator:

- This apparatus is constructed in the latest state of technology and is safe in operation. Nevertheless dangers can arise from the apparatus, especially when it is operated by staff without sufficient training or when it is improperly used or not used as intended.
- Instruct cleaning staff in the cleaning and care of the apparatus from these instructions.
- For translations, the German version of these operating instructions are definitive.
- Changes or alterations made independently to the apparatus are not allowed, for safety reasons.
- If particular problems arise, which are not dealt with fully enough in these operating instructions, please contact your supplier, for your own safety.

Keep these operating instructions carefully, near the apparatus, so as to be able to refer later to safety instructions and important information.

Trade names:

- HANAULUX and *blue 130/100/90* are registered trade names of Maquet.
- All other trade names mentioned in these operating instructions are the sole property of the manufacturer concerned.

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1.1 Explanations of Symbols

Safety symbols draw your attention to operating faults which are critical for safety.

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WARNING!

If not complied with, there may be a severe or even fatal injury.

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CAUTION!

if not complied with, there may be moderate to slight injury or material damage.

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IMPORTANT!

Gives hints for use and useful information.

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ATTENTION !

Comply with assembly and operation instructions

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1.2 General Safety Notes

Please observe the special safety notes in the following sections

1.2.1 Use as intended

- The HANAULUX® *blue 130/100/90* is for illuminating an examination or operation area of a patient in the clinic or doctor's surgery.
- A single *blue 130/100/90* lamp is only suitable for operations in which the light going out does not endanger the patient.
- A HANAULUX *blue 130/100/90* operation light system with several lamps is suitable without limitation.
- The optimum working distance is 70 to 140 cm.
- The HANAULUX *blue 130/100/90* operation is suitable for continuous operation.
- Additional load on the light suspension is not permitted.

1.2.2 Operating Conditions

- The luminaire system is not for operation in areas where there is danger of explosion
- The luminaire system is not suitable for use in combustible mixtures of anaesthetics with air or oxygen or nitrous oxide.
- Keep the ambient temperature from 10°C to 40°C during operation.
- The relative humidity must not exceed 75%.

1.2.3 Requirements for safe operation

The safe and correct function of the luminaire system HANAULUX *blue 130/100/90* within the given technical data is only guaranteed if the following conditions are fulfilled:

- The ceiling anchorage must be statically safesecure and there must be a statics certificate.
- The electrical installations of the rooms concerned must conform to the nationally valid specifications. VDE 0107 applies for Germany.
- Extensions, alterations or repairs must be carried out by Maquet or an authorized specialist.
- The HANAULUX *blue 130/100/90* luminaire system must be assembled as in the assembly instructions accompanying the components.
- For service, repairs and alterations and as accessories, only original Maquet parts may be used.

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1.2.4 Transport and Storage

The following storage conditions apply for up to 15 weeks:

- Temperature -25....+70°C
- Relative humidity 10%....75%
- Air pressure 500hPa....1060hPa

after which the values for the operating conditions apply

- Do not subject the apparatus to severe shaking

1.2.5 Disposal

ENVIRONMENT:

Old units contain materials which are still valuable. Do not take old units to the nearest tip but find out about possible local recycling from the town/ local authority.

- HANAULUX halogen lamps can be disposed of as domestic waste.

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2 Operating the HANAULUX® blue 130/100/90 Luminaire System

2.1 Components Suspension BASIS/LCH (Fig. 1)

HANAULUX® blue 130/100/90 standard components:

- **Canopy (1)** with flange tube and ceiling connection
- **Spacer tube (2)** with central axle
- **Extension arm (3)**
- **Spacer tube (4)**
- **Spring arm (5)**
- **Lamp (7) blue 90**
- **Lamp (8) blue 130**

2.2 Lamp operation

- Turn the lamp into position at the **sterilizable handle (10)** or the **non-sterile edge grip (11)**.
- The optimum working distance is between 70 cm and 140 cm.

In addition, for HANAULUX® blue 90/100:

- **Indicator for reserve lamp (12)** in operation (See section 5.1.1 „lamp changing“)
- Switch the lamp on or off at the **ON/ OFF switch (9)**

In addition, for HANAULUX® blue 130/100 lamp:

- Changing the field size diameter (focussing) by turning the **sterilizable handle (10)**.

In addition, for HANAULUX® blue 130 TV-Dimmer/blue 130 Dimmer lamp (Suspension comfort only)

- Switch the lamp on or off at the **ON/ OFF switch (9)**
- Change the intensity of illumination with the **Plus/ Minus key (13)** (optional). The change of setting is indicated by the four light emitting diodes.

2.3 Action of the Light System

- A special **halogen lamp (14)** acts as light source, with a special filament length matched to the optical system.
- The light bundled in the **reflector (15)** produces a **cascade of light points (16)** below the HANAULUX® blue 130/100/90 lamp.
- An almost **cylindrical light field (17)** is produced, with uniform light distribution in the working area between 70 and 140 cm.
- Depending on the version, a brilliant light of:
blue 90 lamp = 90,000 Lux
blue 100 lamp = 100,000 Lux
blue 130 lamp = 130,000 Lux
is available to you.

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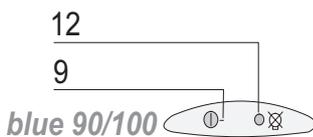
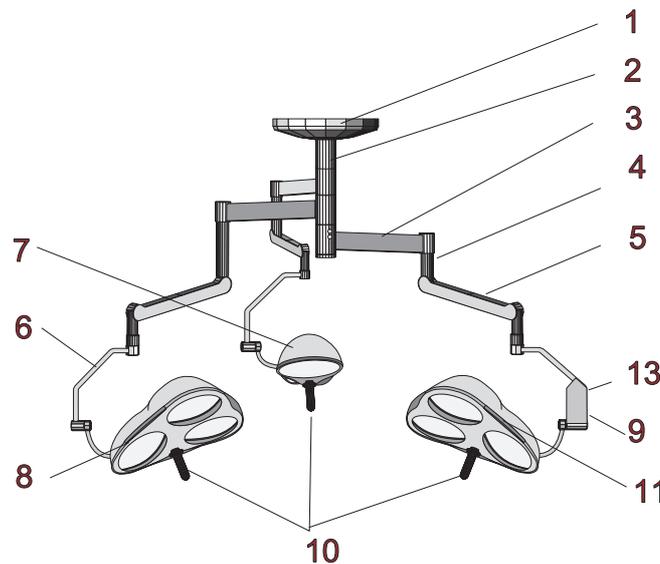
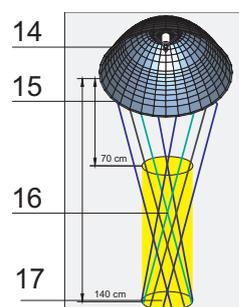
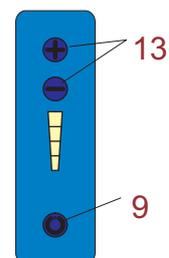
IMPORTANT - Cleaning Staff

Cleaning and disinfection work may be carried out by trained cleaning staff.

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Note:

To avoid damage to plastic and glass parts, do not use any scouring agent, or any cleaning agent which is alkaline, acidic or contains alcohol (e. g. ethanol, propanal, ...) or aldehyde.

- Only disinfect the apparatus when it is cold
- We recommend our tested cleaning/disinfection agent HANAU-clean
- Because there are a great number of cleaning/desinfecting agents on the market, and we do not know their interactions with plastic used, damage cannot be ruled out if other agents are used.
- For cleaning, use a soft, lint-free cloth

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Note

The user has to pay attention to the demands of the national committee for hygiene and disinfection.

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WARNING - Danger to the patients

Please replace immediately damaged handles, which have cracks or are distorted, since these could fall into the wound area.

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- Press the ball retention of the handle inwards and pull the handle off downwards.
- Press the handle on until you hear the ball retention engage and check for secure seating.

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NOTE - Steam sterilization cycles

The handles can withstand at least 350 steam sterilization cycles without damage if the following instructions are observed:

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- Position the handles standing vertically with the open side upwards
- Do not exceed 134 °C sterilization temperature
- Avoid contact of the handles with other objects.

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4.1 Testing

CAUTION - Test Intervals:

All HANAULUX *blue 130/100/90* products are to be tested every half year by the operator on the following points:

- * Damage to paintwork
- * Cracks in plastic parts and closure disc
- * Distortion of carrier system

If any malfunction or damage has occurred, please tell our customer service.

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4.2 Servicing

CAUTION - Service Intervals:

All HANAULUX *blue 130/100/90* products are to be tested/ serviced annually by Maquet or authorized customer service for the following points:

- Testing the action
- Electrical safety test
- Checking the carrier system

Every Service operator is informed and trained by Maquet.

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NOTE - Maintenance staff

Maintenance work may be carried out by one of the operator's service technicians, observing the safety instructions,

WARNING - Electric shock

In all maintenance work, switch off the lighting system at the mains and ensure that it is not switched on again!

5.1 Changing the lamp

WARNING

The halogen bulbs significantly determine the technical data for the HANAULUX operating theatre lights. They are, therefore, accessories under the terms of MDD 93/42/EEC and must therefore only be brought into circulation with the CE symbol as this type of accessory.

Only original Maquet bulbs should be used as any of the following could otherwise occur:

- A lower light-intensity
- A higher temperature increase in the operating field
- Inhomogenous lightfield
- Reduced bulb life
- Damage to the optical system.

5.1.2 Lamp blue 130 (Fig. 2)

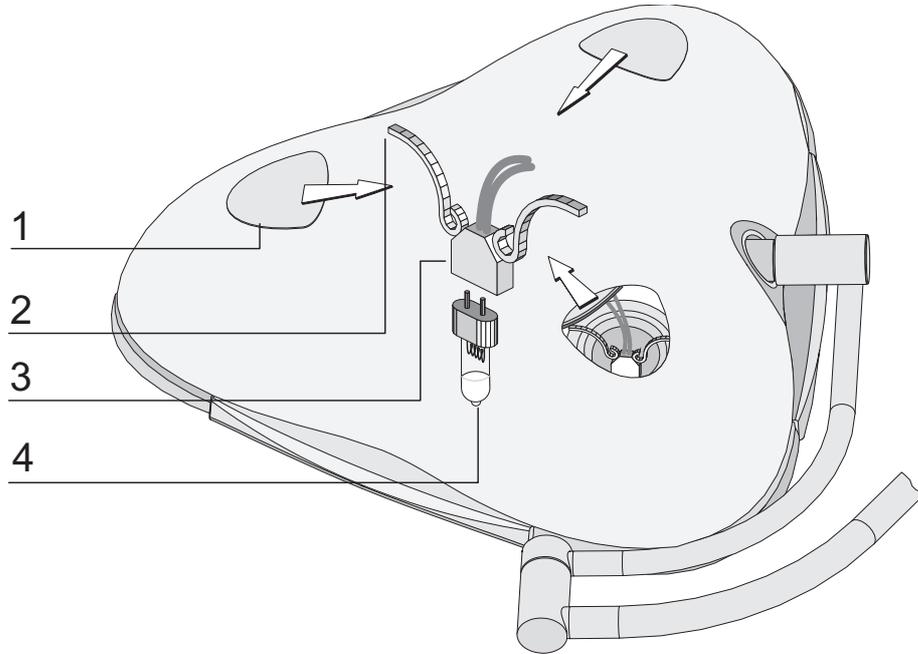
1. Press in **cover (1)** until you feel it meet the stop and push it to the side in the direction of the arrow.
2. Pull out the **socket (2)** between the **spring clips (3)**.
3. Take out the **defective lamp (4)** and put a new halogen lamp in (wear gloves).
4. Put the **socket (2)** into the reflector until the **spring clips (3)** engage.
5. Push the **cover (1)** back until it engages.

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5.1.2 Lamp blue 100 (Fig. 3)

 **CAUTION - Reserve lamp in operation:** 

If this signal shows in the operator's control field of the *blue 100* lamp, the reserve lamp is in operation. To avoid complete non-operation of the light system, the main lamp should be changed as soon as possible. The position of the reserve lamp is shown by an R on the double lamp socket.

1. Press in **cover (1)** until you feel it meet the stop and only then push it to the side in the direction of the arrow.
2. Turn the **socket fixing (2)** to the side.
3. Pull out the **socket (4)**
4. Take out the **defective main lamp (5)**
5. Take out the **reserve lamp (6)** marked R and place it in the **socket of the main lamp (7)** (lamp exchange).
6. Put a new halogen lamp in the **reserve lamp position (8)** (wear gloves).

5.1.3 Lamp blue 90 (Fig. 4)

 **CAUTION - Reserve lamp in operation:** 

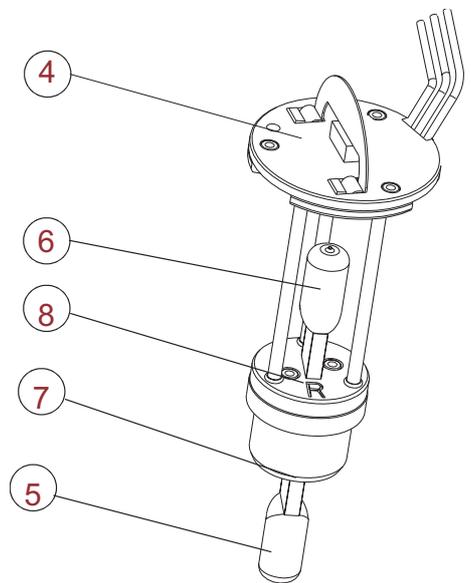
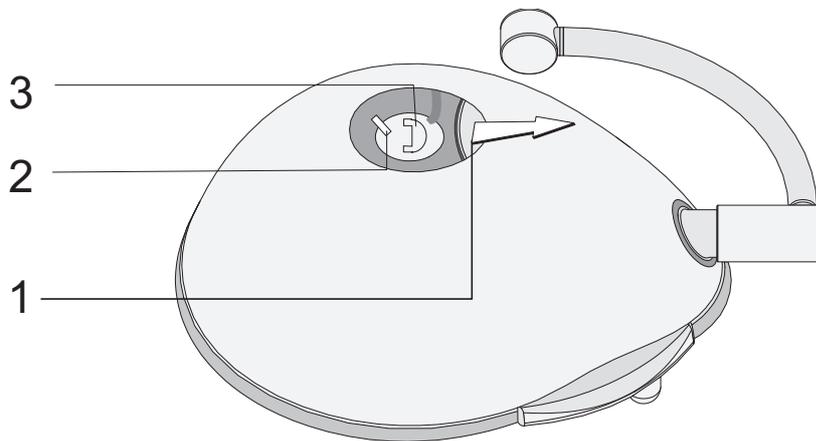
If this signal shows in the operator's control field of the *blue 90* lamp, the reserve lamp is in operation. To avoid complete non-operation of the light system, the main lamp should be changed as soon as possible. The position of the reserve lamp is shown by an R on the double lamp socket.

1. Press in **cover (1)** until you feel it meet the stop and only then push it to the side in the direction of the arrow.
2. Pull out the **socket (2)** between the **spring clips (3)**.
3. Take out the **defective main lamp (4)** on the side of the **supply leads (5)**.
4. Take out the **reserve lamp (6)** marked R and place it in the socket of the main lamp (lamp exchange).
5. Put a new halogen lamp in the reserve lamp position (wear gloves).
6. Put the **socket (2)** into the reflector until the **spring clips (3)** engage.
7. Push the **cover (1)** back until it engages and check that the lid is seated securely.

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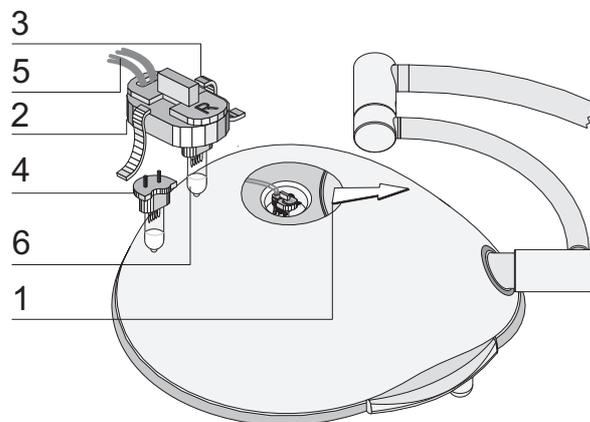
IMPORTANT - Maintenance staff

Maintenance work may be carried out by one of the operator's service technicians, observing the safety instructions.

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IMPORTANT:

If the lamp does not automatically stay in place in every position, the spring force of the spring arm must be increased or decreased.

1. Insert allen key (supplied by spring arm) fully into the **opening (1)**.
2. Push the spring-arm upwards
3. Turning allen key to the right = reduces the spring-force.
Turning to the left = Increases the spring-force.

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NOTE :

The brakes are set on installation. Like all mechanical parts, the brakes are subject to abrasion.

If the lamp does not automatically stay in place in every position, the brake force of the brake screws must be adjusted.

1. The braking effect can be changed by tightening or slackening the **brake screws (1-6)**.

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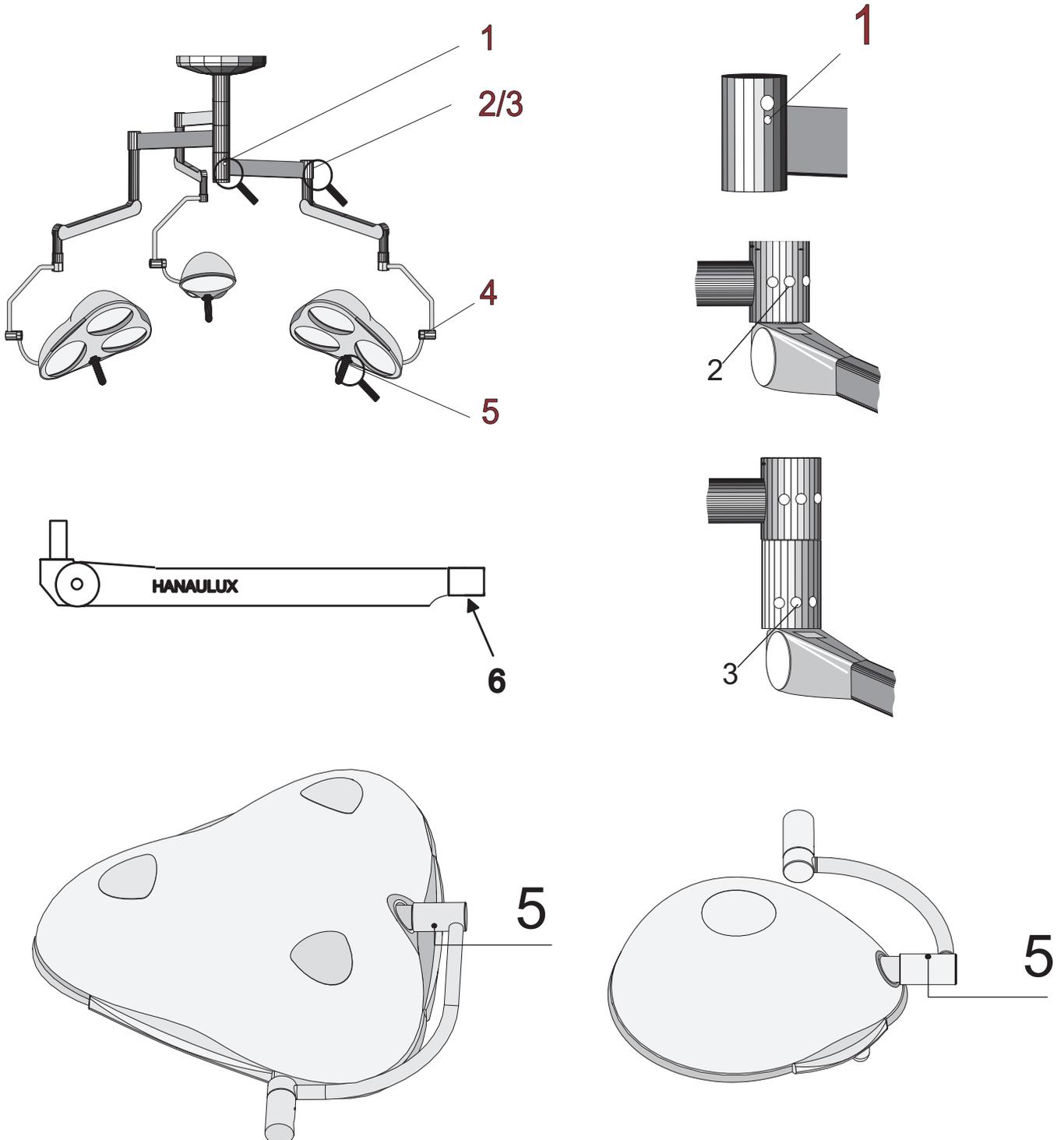
- 1 Brake screw central axle
- 2 Brake screw spring arm LCH (Main light)
- 3 Brake screw spring arm (If assembled to spacer)
- 4 Brake screw comfort bracket
- 5 Brake screw light head
- 6 Brake screw spring arm LCH

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6 Troubleshooting

Fault	Possible cause	Remedy	Reference
-------	----------------	--------	-----------

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Light is heavy to move	Brakes are set too tight	Set brakes	Sect. 5.3
Light moves too easily	Brakes are set too loose	Set brakes	Sect. 5.3
Lamp falls/ rises	Spring force in weight counterbalance / spring arm too small/ too large	Set spring force	Sect. 5.2

Optical system/ light mechanism

Light flickers	Kontaktprobleme	Service	---
Light intensity too bright/ too dark	Wrong supply voltage	Service	---
Lamp doesn't last long enough	Wrong lamp(s) Wrong supply voltage	Service	---
Light field not round	Wrong lamp(s) Lamp seating incorrect Main lamp defective	Check lamp and if necessary replace it Check lamp seating	Sect/ 5.1

Closure disc

Scratched or cracked closure disc	Incorrect cleaning agent used, Cleaning or disinfecting when hot	Use different cleaning agent	Sect. 3.1
-----------------------------------	---	------------------------------	-----------

Sterilization

Sterilizable handle has too short a life	Wrong sterilization procedure	Check sterilization procedure	Sect. 3.2
Handles damaged or having cracks	Ende der Nutzungsdauer erreicht	Replace handles	Sect. 3.2

If a fault occurs repeatedly or cannot be remedied, please get in touch with your Maquet contact partner.

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7 Technical Data

Technical light data	HANAULUX <i>blue 90</i>	HANAULUX <i>blue 100</i>	HANAULUX <i>blue 130</i>
Illuminance E_c	90.000 Lux	100.000 Lux	130.000 Lux
Light field diameter d_{10}	16 cm	18-23 cm	20 - 32 cm
Light field diameter d_{50}	9 cm	9 cm	10 cm
Remaining illuminance one mask	-	-	70%
Remaining illuminance two mask	50%	48%	50%
Remaining illuminance / tube	100%	100%	90%
Remaining illuminance tube and one mask	-	-	55%
Remaining illuminance tube and two mask	50%	48%	45%
Working distance	70 - 140 cm	70-140 cm	70 - 140 cm
Temperature rise in operation field at 100 klx	15 °C	15 °C	14 °C
Luminous efficiency	250 lm/W	250 lm/W	270 lm/W
Total irradiance (at nominal illuminance)	360 W/m ²	400 W/m ²	481 W/m ²
Colour temperature	4.300 K	4300 K	4.300 K
IEC 601-2-41 is the basis of all technical light data			
Electrical data	HANAULUX <i>blue 90</i>	HANAULUX <i>blue 100</i>	HANAULUX <i>blue 130</i>
Supply voltage (terminal)	24 V DC 24 V AC 50-60 Hz	24 V DC 24 V AC 50-60 Hz	24 V DC 24 V AC 50-60 Hz
Mechanical data	HANAULUX <i>blue 90</i>	HANAULUX <i>blue 100</i>	HANAULUX <i>blue 130</i>
Height of movement from highest to lowest lamp position: LCH suspension	approx. 1140 mm	approx. 1140 mm	approx. 1260 mm
Height of movement from highest to lowest lamp position: Basis/Comfort suspension	approx. 1180 mm	approx. 1180 mm	approx. 1180 mm
Classification	HANAULUX <i>blue 90</i>	HANAULUX <i>blue 100</i>	HANAULUX <i>blue 130</i>
Protection class	I	I	I
Type of operation	continuous	continuous	continuous
Guideline of Council 33/42 EEC- MPG	Class 1	Class 1	Class 1

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