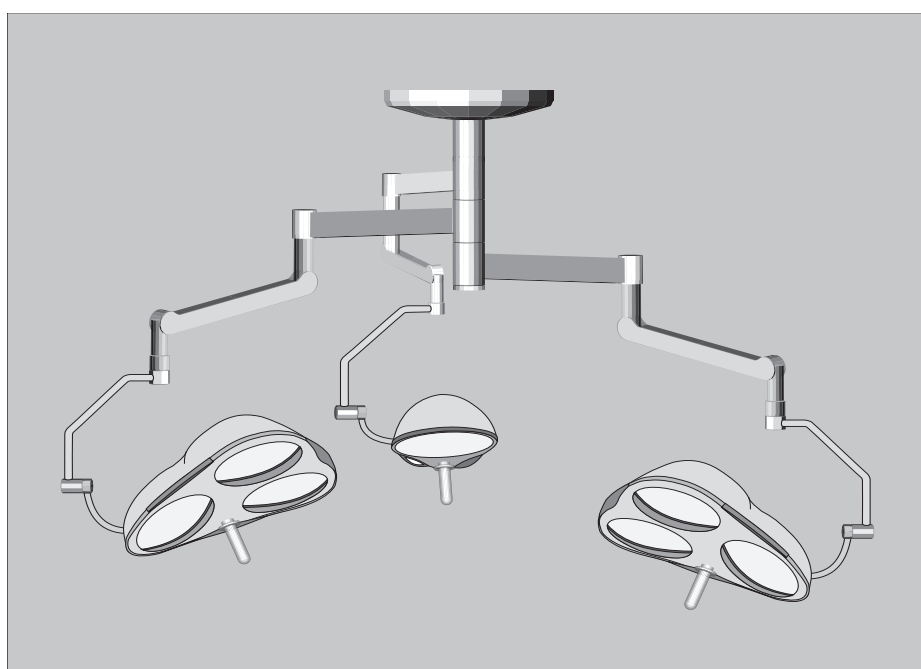
BLUE 130/90

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Operating instructions

56 351 001/F

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Contents	Page
1 Safety advice	4
1.1 Explanation of symbols	5
1.2 General safety instructions.....	6
2 Operating the <i>BLUE 130/90</i> lighting system	8
2.1 Components of the lighting system	8
2.2 Positioning the lighthouse	8
2.3 Functions of the lighting system	8
3 Cleaning and disinfection	10
3.1 Cleaning/Disinfection	10
3.2 Sterilizing the handles	10
4 Testing and servicing	11
4.1 Testing.....	11
4.2 Servicing	11
5 Maintenance	12
5.1 Changing the light bulb	12
5.1.1 Changing the <i>BLUE 130</i> light bulb	12
5.1.2 Changing the <i>BLUE 90</i> light bulb	14
5.2 Adjusting the spring force.....	16
5.4 Adjusting the brake force	18
5.5 Consumable material	18
6 Fault finding	20
7 Technical data	22

Figures	Page
Fig. 1 Operating the <i>BLUE 90/ 130</i> lighting system	9
Fig. 2 Changing the <i>BLUE 130</i> light bulb.....	13
Fig. 3 Changing the <i>BLUE 90</i> light bulb.....	15
Fig. 4 Adjusting the spring force.....	17
Fig. 5 Adjusting the brake force <i>BLUE 130/90</i>	19

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Dear user:

- These instructions for use are intended for the medical and technical staff of the hospital or the doctor's practise.
- The controls in the "Maintenance" section may be carried out by the facility's service technicians, who must follow the safety instructions,
- The cleaning of the apparatus may be carried out by trained cleaning staff.

These operating instructions apply to all **BLUE 90/ 130** ceiling models.

- Please read these operating instructions carefully before using the apparatus. You will then enjoy all the advantages the apparatus offers and protect yourself and others from harm.

Dear Operator:

- This apparatus is constructed in the latest state of technology and is safe in operation. Nevertheless dangers can arise from the apparatus, especially when it is operated by staff without sufficient training or when it is improperly used or not used as intended.
- Instruct cleaning staff in the cleaning and care of the apparatus from these instructions.
- For translations, the German version of these operating instructions are definitive.
- Changes or alterations made independently to the apparatus are not allowed, for safety reasons.
- If particular problems arise, which are not dealt with fully enough in these operating instructions, please contact your supplier, for your own safety.

Keep these operating instructions carefully, near the apparatus, so as to be able to refer later to safety instructions and important information.

Trade names:

- **BLUE 130/90** is a registered trade name of MAQUET.
- All other trade names mentioned in these operating instructions are the sole property of the manufacturer concerned.

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1.1 Explanations of Symbols

Safety symbols draw your attention to operating faults which are critical for safety.

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WARNING!

If not complied with, there may be a severe or even fatal injury.

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CAUTION!

if not complied with, there may be moderate to slight injury or material damage.

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IMPORTANT!

Gives hints for use and useful information.

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ATTENTION !

Comply with assembly and operation instructions

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1.2 General Safety Notes

Please observe the special safety notes in the following sections

1.2.1 Use as intended

- The **BLUE 130/90** is for illuminating an examination or operation area of a patient in the clinic or doctor's surgery.
- A single **BLUE 130/90** lighthouse is only suitable for operations in which the light going out does not endanger the patient.
- A **BLUE 130/90** operation lighting system with several lamps is suitable without limitation.
- The optimum working distance is 70 to 140 cm.
- The **BLUE 130/90** operation light is suitable for continuous operation.
- Additional load on the light suspension is not permitted.

1.2.2 Operating Conditions

- The lighting system is not for operation in areas where there is danger of explosion
- The lighting system is not suitable for use in combustible mixtures of anaesthetics with air or oxygen or nitrous oxide.
- Keep the ambient temperature from 10°C to 40°C during operation.
- The relative humidity must not exceed 75%.

1.2.3 Requirements for safe operation

The safe and correct function of the lighting system **BLUE 130/90** within the given technical data is only guaranteed if the following conditions are fulfilled:

- The ceiling anchorage must be statically safe and there must be a static certificate.
- The electrical installations of the rooms concerned must conform to the nationally valid specifications. VDE 0107 applies for Germany.
- Extensions, alterations or repairs must be carried out by MAQUET or an authorized specialist.
- The **BLUE 130/90** lighting system must be assembled as in the assembly instructions accompanying the components.
- For service, repairs and alterations and as accessories, only original MAQUET parts may be used.

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1.2.4 Transport and Storage

The following storage conditions apply for up to 15 weeks:

- Temperature -25 to + 70°C
- Relative humidity 10% to 75%
- Air pressure 500hPa to 1060hPa

after which the values for the operating conditions apply

- Do not subject the apparatus to severe shaking

1.2.5 Disposal

ENVIRONMENT:

Old units contain materials which are still valuable. Do not take old units to the nearest tip but find out about possible local recycling from the town/ local authority.

-

- BLUE halogen lamps can be disposed of as domestic waste.

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2 Operating the *BLUE 130/90* lighting system

2.1 Components of the *BASIS/LCH* Suspension (Fig. 1)

BLUE 130/90 standard components:

- **Canopy (1)** with flange tube and ceiling connection
- **Spacer tube (2)** with central axle
- **Extension arm (3)**
- **Spacer tube (4)**
- **Spring arm (5)**
- ***BLUE 90* lighthouse (7)**
- ***BLUE 130* lighthouse (8)**

2.2 Positioning the lighthouse

- Turn the lighthouse into position using the **sterilizable handle (10)** or the **non-sterile edge grip (11)**.
- The optimum working distance is between 70 cm and 140 cm.

In addition, for *BLUE 90:*

- **Indicator that reserve light bulb (12) is** in operation (See section 5.1.1 "Changing the light bulb")
- Switch the light bulb on or off at the **ON/ OFF switch (9)**

In addition, for *BLUE 130* lighthouse:

- Change the field size diameter (focussing) by turning the **sterilizable handle (10)**.

In addition, for *BLUE 130 TV-Dimmer/BLUE 130 Dimmer lighthouse (Comfort Suspension only)*

- Switch the lighthouse on or off at the **ON/ OFF switch (9)**
- Change the intensity of illumination with the **Plus/Minus key (13)** (optional). The change of setting is indicated by the four light emitting diodes.

2.3 Functions of the lighting system

- A special **halogen light bulb (14)** acts as light source, with a special filament length matched to the optical system.
- The light bundled in the **reflector (15)** produces a **cascade of light points (16)** below the *BLUE 130/90* lighthouse.
- An almost **cylindrical light field (17)** is produced, with uniform light distribution in the working area between 70 and 140 cm.
- Depending on the version, the lighthouse produces the following illuminances:
BLUE 90 lighthouse = 90,000 Lux
BLUE 130 lighthouse = 130,000 Lux

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
2 Operating the *BLUE 130/90* lighting system

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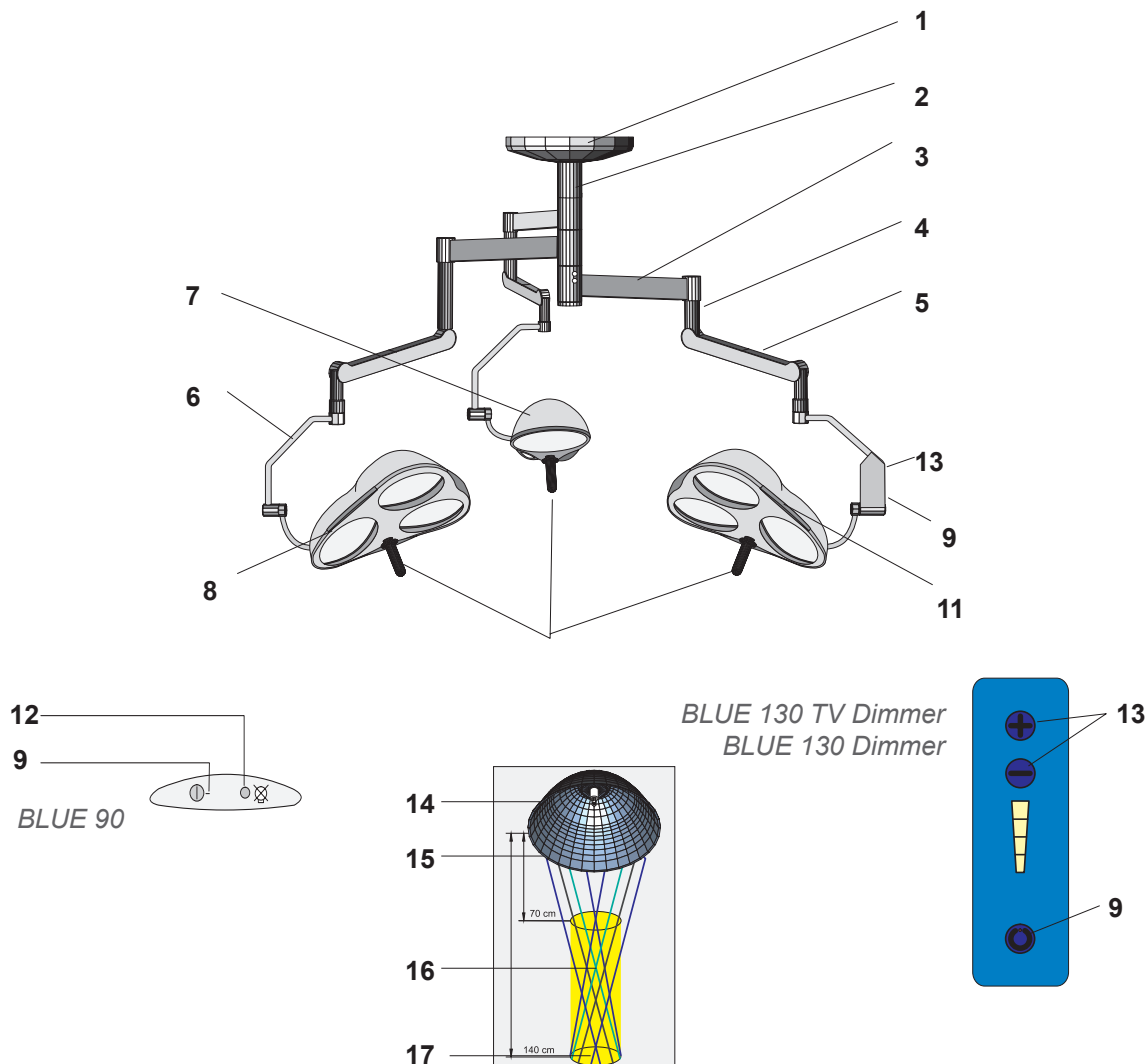IMPORTANT - Cleaning Staff

Cleaning and disinfection work may be carried out by trained cleaning staff.

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Note:

To avoid damage to plastic and glass parts, do not use any scouring agent, or any cleaning agent which is alkaline, acidic or contains alcohol (e. g. ethanol, propanol, ...), aldehyde or chlorine.

- Only disinfect the apparatus when it is cold.
- Because there are a great number of cleaning/desinfecting agents on the market, and we do not know their interactions with plastic used, damage cannot be ruled out if other agents are used.
- For cleaning, use a soft, lint-free cloth.

Note

The user must comply with national requirements concerning hygiene and disinfection.

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WARNING - Danger to the patients

Please replace immediately damaged handles, which have cracks or are distorted, since these could fall into the wound area.

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- Press the button on the handle and pull the handle off downwards.
- Press the handle on until it clicks into position and check for secure seating.

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NOTE - Steam sterilization cycles

The handles can withstand at least 350 steam sterilization cycles without damage if the following instructions are observed:

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- Place the handles straight with the opening on top.
- Do not exceed 134 °C sterilization temperature
- Avoid contact of the handles with other objects.

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4.1 Testing

 **CAUTION - Test Intervals:**
The following points must be tested every 6 months on all **BLUE 130/90** products:

- * Damage to paintwork
- * Cracks in plastic parts and closure disc
- * Distortion of suspension system

If any malfunction or damage has occurred, please tell our customer service.

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4.2 Servicing

 **CAUTION - Service Intervals:**
The following points must be tested once a year by MAQUET or authorized customer service on all **BLUE 130/90** products:

- Testing functions
- Electrical safety test
- Checking the suspension system
- Only for **BLUE 130**: Check the front pivot on Acrobat 2000 spring arms (if manufactured between 2000 and 2006). Change the spring arm if any sign of crack is seen.

Every Service operator must be informed and trained by MAQUET.

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NOTE - Maintenance staff

Maintenance must be carried out by one of the facility's service technicians, according to the safety instructions.

WARNING - Electric shock

For all maintenance work, switch off the lighting system at the mains and ensure that it is not switched on again!

5.1 Changing the light bulb

WARNING

The halogen bulbs significantly determine the technical characteristics of the BLUE surgical lights. They are, therefore, covered by the MDD 93/42/EEC directive and must carry the EC symbol to be put on the market. Only original MAQUET bulbs should be used as any of the following could otherwise occur:

- Reduced light intensity
- A higher temperature in the operating field
- Inhomogenous lightfield
- Reduced bulb life
- Damage to the optical system.

5.1.2 BLUE 130 lighthead (Fig. 2)


1. Press in **cover (1)** until you feel it meet the stop and push it in the direction of the arrow.
2. Pull out the **socket (2)** between the **spring clips (3)**.
3. Take out the **defective light bulb (4)** and put a new halogen light bulb in (wear gloves).
4. Put the **socket (2)** into the reflector until the **spring clips (3)** engage.
5. Push the **cover (1)** back until it engages.

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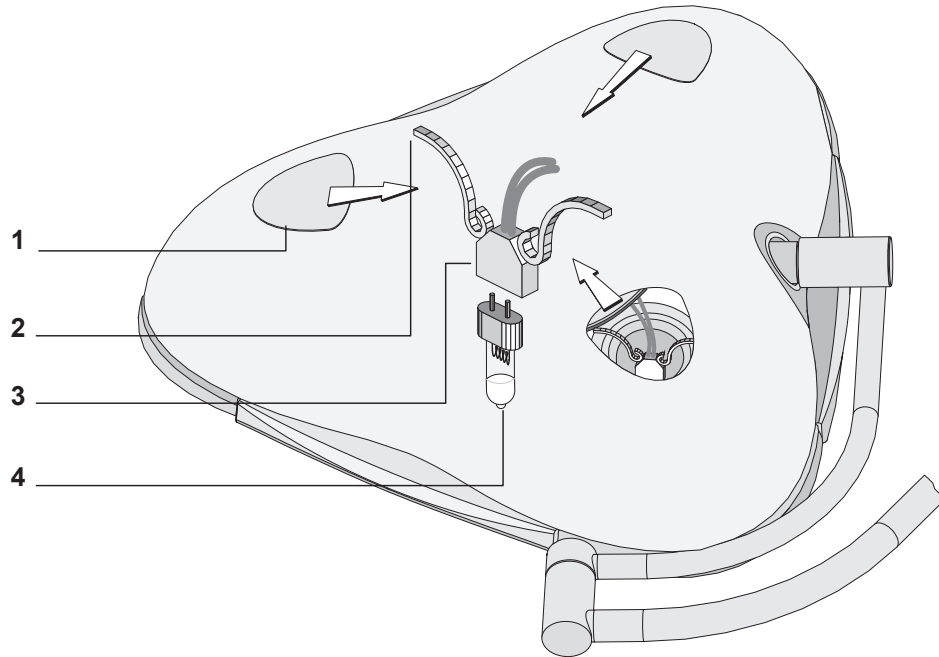- 1 **Cover**
- 2 **Spring clips**
- 3 **Socket**
- 4 **Light bulb**

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5.1.2 BLUE 90 lighthouse (Fig. 3)

 **CAUTION** - Reserve light bulb in operation:

If this signal shows in the operator's control field of the **BLUE 90** lighthouse, the reserve light bulb is in operation. To avoid complete non-operation of the lighting system, the main light bulb should be changed as soon as possible. The position of the reserve light bulb is shown by an **R** on the double light bulb socket.


1. Press in **cover (1)** until you feel it meet the stop and only then push it to the side in the direction of the arrow.
2. Pull out the **socket (2)** between the **spring clips (3)**.
3. Take out the **defective main light bulb (4)** on the side of the **supply leads (5)**.
4. Take out the **reserve light bulb (6)** marked **R** and place it in the socket of the main light bulb (light bulb exchange).
5. Put a new halogen light bulb in the reserve light bulb position (wear gloves).
6. Put the **socket (2)** into the reflector until the **spring clips (3)** engage.
7. Push the **cover (1)** back until it engages and check that the lid is seated securely.

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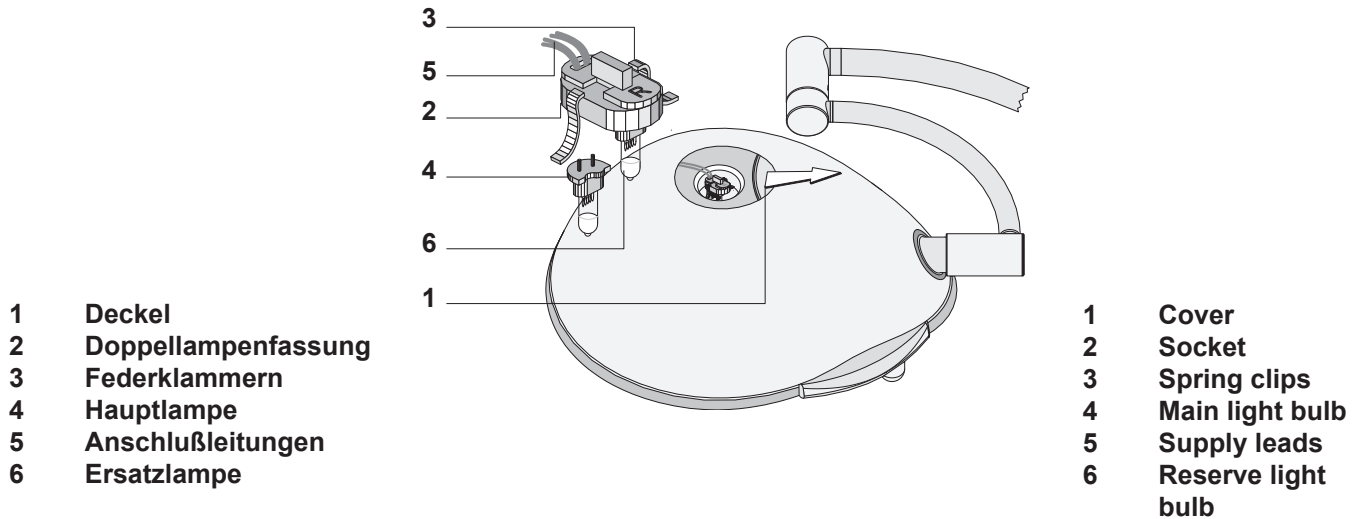IMPORTANT - Maintenance staff

Maintenance must be carried out by one of the facility's service technicians, according to the safety instructions..

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IMPORTANT:

If the lighthouse does not remain in every position, the spring force of the spring arm must be increased or decreased.


1. Insert allen key (supplied by spring arm) fully into the **opening (1)**.
2. Push the spring-arm upwards
3. Turning allen key to the right = reduces the spring-force.
Turning to the left = Increases the spring-force.

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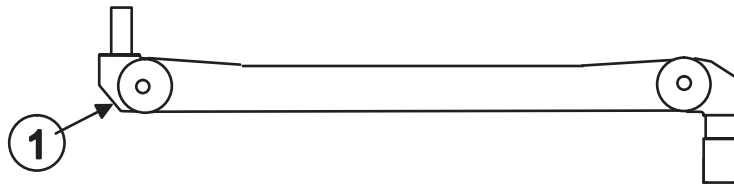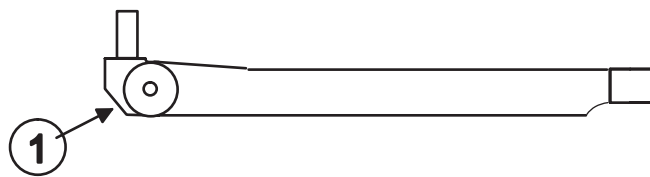
5.3 Adjusting the brake force (Fig 5)

NOTE :

The brakes are set on installation. Like all mechanical parts, the brakes are subject to abrasion.

If the lighthouse does not automatically remain in every position, the brake force of the brake screws must be adjusted.

1. The braking effect can be changed by tightening or slackening the **brake screws (1-6)**.

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5.4 Consumable material

CAUTION

The halogen bulbs significantly determine the technical characteristics of the BLUE surgical lights. They are, therefore, covered by the MDD 93/42/EEC directive and must carry the EC symbol to be put on the market. Only original MAQUET bulbs should be used as any of the following could otherwise occur:

- Reduced light intensity
- A higher temperature increase in the operating field
- Inhomogenous lightfield
- Reduced bulb life
- Damage to the optical system.

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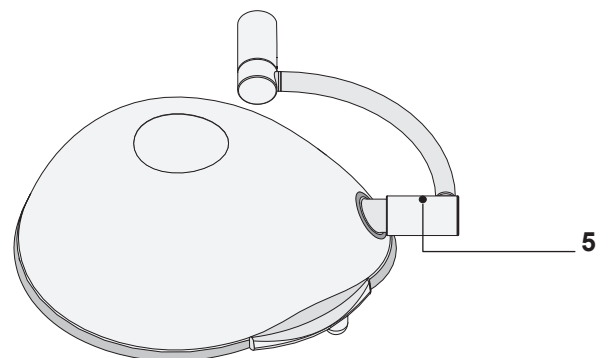
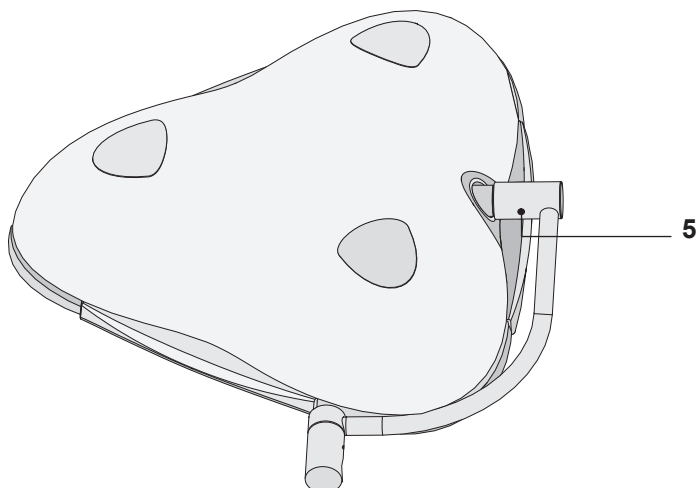
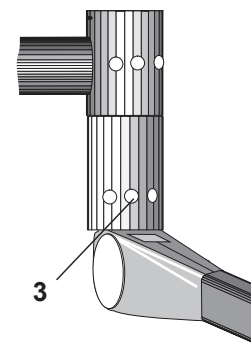
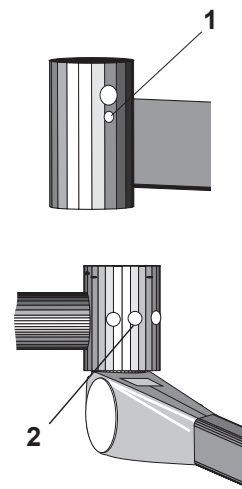
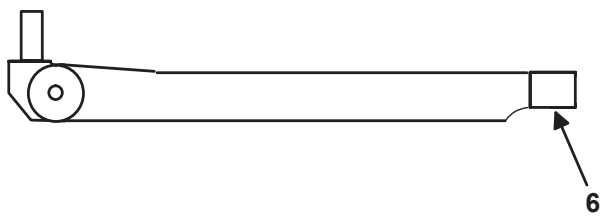
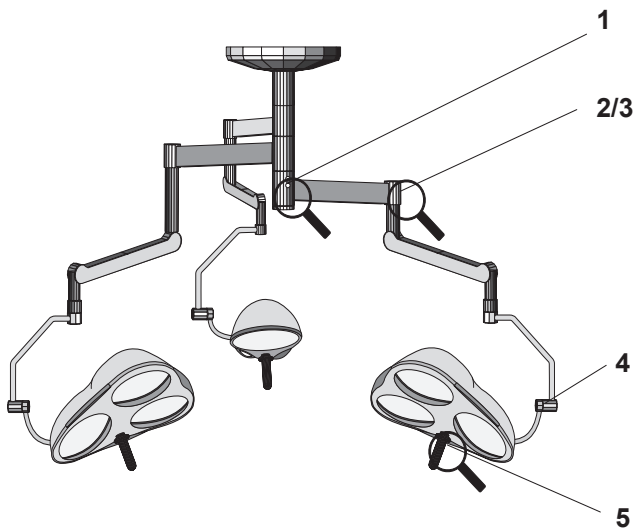
- 1 Brake screw central axle
- 2 Brake screw spring arm LCH (Main light)
- 3 Brake screw spring arm (If assembled to spacer)
- 4 Brake screw comfort bracket
- 5 Brake screw light head
- 6 Brake screw spring arm LCH

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6 Troubleshooting

Fault	Possible cause	Remedy	Reference
-------	----------------	--------	-----------

Suspension / Mobility

Light is heavy to move	Brakes are set too tight	Set brakes	Sect. 5.3
Light moves too easily	Brakes are set too loose	Set brakes	Sect. 5.3
Lighthouse falls/ rises	Spring force in weight counterbalance / spring arm too small/ too large	Set spring force	Sect. 5.2

Optical system/ light mechanism

Light flickers	Contact problem	Service	---
Light intensity too bright/ too dark	Wrong supply voltage	Service	---
Light bulb doesn't last long enough	Wrong light bulb(s) Wrong supply voltage	Service	---
Light field not round	Wrong light bulb(s) Light bulb seating incorrect Main light bulb defective	Check light bulb and if necessary replace it Check light bulb seating	Sect/ 5.1

Closure disc

Scratched or cracked closure disc	Incorrect cleaning agent used, Cleaning or disinfecting when hot	Use different cleaning agent	Sect. 3.1
-----------------------------------	---	------------------------------	-----------

Sterilization

Sterilizable handle has too short a life	Wrong sterilization procedure	Check sterilization procedure	Sect. 3.2
Handles damaged or having cracks	End of handles' life cycle	Replace handles	Sect. 3.2

If a fault occurs repeatedly or cannot be remedied, please get in touch with your MAQUET contact partner.

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7 Technical Data

Technical light data	<i>BLUE 90</i>	<i>BLUE 130</i>
Illuminance E_c	90.000 Lux	130.000 Lux
Light field diameter d_{10}	16 cm	20 - 32 cm
Light field diameter d_{50}	9 cm	10 cm
Remaining illuminance one mask	-	70%
Remaining illuminance two mask	50%	50%
Remaining illuminance / tube	100%	90%
Remaining illuminance tube and one mask	-	55%
Remaining illuminance tube and two mask	50%	45%
Working distance	70 - 140 cm	70 - 140 cm
Temperature rise in operation field at 100 klx	15 °C	14 °C
Luminous efficiency	250 lm/W	270 lm/W
Total irradiance (at nominal illuminance)	360 W/m ²	481 W/m ²
Colour temperature	4.300 K	4.300 K
IEC 601-2-41 is the basis of all technical light data		
Electrical data	<i>BLUE 90</i>	<i>BLUE 130</i>
Supply voltage (terminal)	24 V DC 24 V AC 50-60 Hz	24 V DC 24 V AC 50-60 Hz
Mechanical data	<i>BLUE 90</i>	<i>BLUE 130</i>
Height of movement from highest to lowest lamp position: LCH suspension	approx. 1140 mm	approx. 1260 mm
Height of movement from highest to lowest lamp position: Basis/Comfort suspension	approx. 1180 mm	approx. 1180 mm
Classification	<i>BLUE 90</i>	<i>BLUE 130</i>
Protection class	I	I
Type of operation	continuous	continuous
Guideline of Council 33/42 EEC- MPG	Class 1	Class 1

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