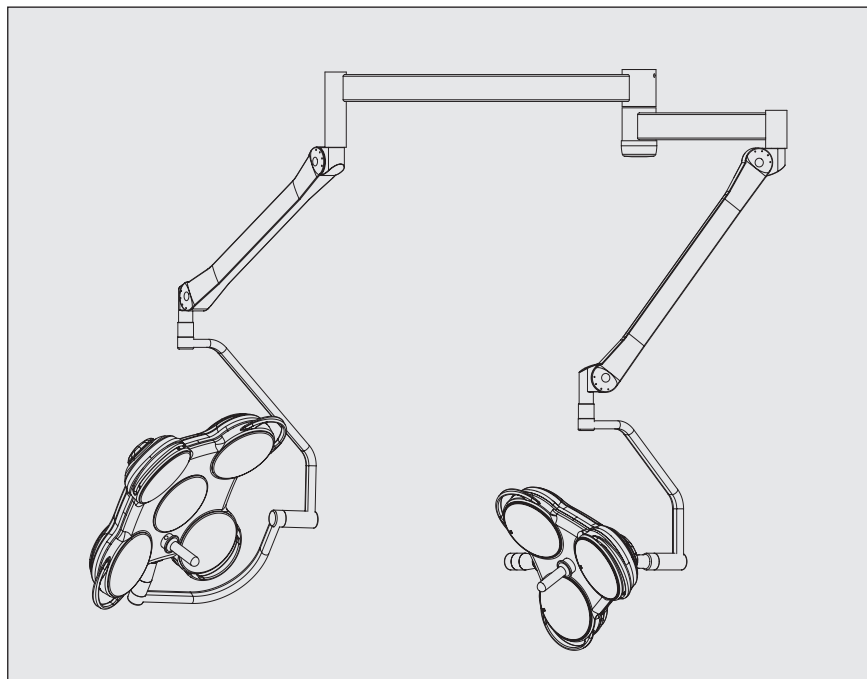
# HLX 3000

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*Manual técnico* **ES**

**CONTENTS**

<b>QUALITY COMPLIANCE</b> .....	<b>4</b>
<b>1 GENERAL CHARACTERISTICS</b> .....	<b>5</b>
<b>2 MAINTENANCE</b> .....	<b>6</b>
2.1 Preventive maintenance.....	6
2.2 First level maintenance.....	6
<b>3 CLEANING/DISINFECTION/STERILIZATION</b> .....	<b>8</b>
3.1 Cleaning and disinfecting the surgical light.....	8
3.2 Cleaning and sterilizing the handles.....	8
<b>4 MAINTENANCE KITS</b> .....	<b>10</b>
4.1 Recommended spare parts and consumables.....	12
4.2 Maintenance tools (OPM).....	12
<b>5 MAINTENANCE PROCEDURES</b> .....	<b>13</b>
5.1 Disassembling a cover.....	13
5.2 Disassembling a hinge.....	13
5.3 Disassembling the upper shell.....	13
5.4 Disassembling the ergofocus.....	13
5.5 Disassembling the side grab handles.....	14
5.6 Disassembling a lens.....	15
5.7 Disassembling a reflector.....	15
5.8 Disassembling a bulb holder.....	15
<b>6 PARTS LISTS AND DIAGRAMS</b> .....	<b>16</b>
<b>7 FORKS</b> .....	<b>42</b>
7.1 Oblongs forks.....	42
7.2 Round forks.....	44
<b>8 SPRING ARMS</b> .....	<b>45</b>
<b>9 TROUBLESHOOTING</b> .....	<b>46</b>

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## QUALITY STANDARDS COMPLIANCE

### Certification of MAQUET SAS's quality system

LNE/G-MED certifies that the quality system developed by MAQUET SAS for design, implementation, sales, installation and after-sales service of surgical lights complies with the requirements of the following international standards:

- ISO 9001:2008
- NF EN ISO 13485:2012.

### CE Marking/Intended use

Compliance with the requirements of Directive 93/42/EEC dated of June 14th 1993, relating to medical devices has been assessed in accordance with Annex VII of this Directive. The HLX 3000 surgical light is a class I device in accordance with Annex IX of Directive 93/42/EEC.

The range includes single, double or triple ceiling configurations, with or without video pre-equipment (HANAUVISION), and the following possible options:

- confort bracket
- flat screen support
- separate camera support.

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## 2 MAINTENANCE

### 2.1 Preventive maintenance

To preserve your surgical lights' original performance and reliability, annual maintenance and inspections should be performed as follows:

- during the warranty period, by a MAQUET technician or a MAQUET-certified distributor,
- outside the warranty period, by a MAQUET technician or a MAQUET-certified distributor, or the hospital's technical maintenance department.

### 2.2 First level maintenance

#### Permanent maintenance

Perform following checks:

- Check for presence and readability of identification labels.
- Check general condition of arm covers.
- Check condition of lenses.
- Check tightness of joints.
- Check cleanliness of suspension arms.

#### 2.2.1 Daily inspection:

- Check that bulbs operate correctly.
- Check that the sterilizable handle clicks and locks in correctly.
- Check arm's position.

#### 2.2.2 Monthly inspection:

- Check that the lamp operates correctly from the backup power supply should a mains power supply interruption occur.
- Check presence of end stops.
- Check that the light switches off when the cupola is turned upwards.

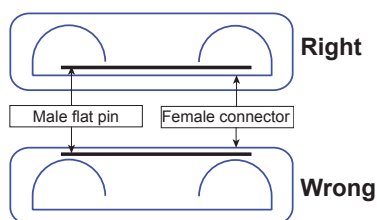
#### 2.2.3 Annual inspection (to be done by an authorized technician):

- Check the lamp socket.
- Check presence of end stops on flat screen arms and on the cupolas.

#### Safety points

Check the following:

- Every six years, as a preventive measure, replace the mounting screws holding the suspension tube to the anchor point with new pre-glued screws, and tighten to torque (refer to the appendix in the technical manual).
- Check that the seals are correctly located.
- Installation of the arm(s).
- Additional cupola's mounting stop segments, removal of cupolas and lubrication of sleeve.
- Attachment of all covers and caps.

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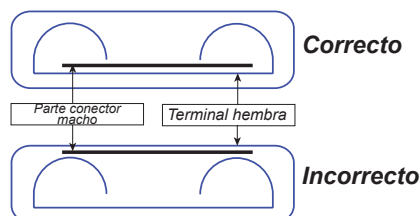
- Check the terminal/switch connection:
  - The male terminal (switch) should be centred in the female terminal (cable)
- Check the tightness of the locknut on the suspensions without stops.
- Only for HLX 3004/3005: Check the front pivot on Acrobat 2000 spring arm (if manufactured between 2004 and 2006). Change the spring arm if any sign of crack is seen.

#### Other checks

- Nominal illumination: see technical data.
- Continuity of grounds 0.1 Ohm max.
- State of Fresnel lenses and thermal filter.
- Verticality of suspension tube.
- Balancing adjustment.
- Locking mechanism of sterilizable handle.
- In START UP menu, check the software version of the Microcontroller Board. If the software version is before 5.40 (2.05;2.06 for example), inform the customer that this version is obsolete. See NIT 188 for more informations.

Dismantling certain elements can affect operation and safety, for example:

- When servicing the electrical power supply,
- When servicing the suspension arms and balancing system,
- When servicing the optical system of the cupolas which are equipped with filters designed to eliminate radiations not visible by the patient. The unit should not be operated, under any circumstances, without these filters.

For this type of service, contact the MAQUET-approved product support service.

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### 3 CLEANING / DISINFECTION / STERILISATION

Users must contact their hospital's sanitary specialists. The recommended products and procedures must be applied. Should there be any doubt concerning the compatibility of active agents to be used, contact the local MAQUET customer service.

#### 3.1 Cleaning and disinfecting the surgical light

 Check that the power is switched off and the light has cooled down before starting cleaning.

##### 3.1.1 General instructions concerning cleaning, disinfection and safety:

- Remove the sterilisable handles.
- Wipe the system with a cloth moistened with a surface cleaner. Follow the manufacturer's dilution and temperature recommendations.
- Rinse with a cloth moistened with water. Wipe dry.
- Wipe evenly with a cloth moistened with disinfectant. Follow the manufacturer's recommendations.
- Rinse with a cloth moistened with water to remove any residue (particularly when cleaners containing aldehydes, quaternary ammonium or surfactants are used).
- Wipe off with a dry cloth.
- Make sure that all liquid cleaning products used have been thoroughly wiped off.

##### 3.1.2 Examples of recommended products

**GETINGE USA products:** TEC-QUAT 256.

**ANIOS products:** SURFA'SAFE; HEXANIOS G + R at 0.5%; ANIOSYME P.L.A.; SALVANIOS pH10; ANIOS DDSH.

**Schülke & Mayr products:** ANTIFECT PLUS.

##### 3.1.3 Examples of prohibited products

 Solutions containing glutaraldehyde, phenol, iodine, bleach, alcohol or chloride ions must not be used.

Do not use disinfection by fumigation methods.

#### 3.2 Cleaning and sterilising the handles

##### 3.2.1 Before cleaning

- Use a soft cloth immediately after use to wipe away soiling from the handle surface.
- Store handles in a place that keeps them moist to make further cleaning easier.
- Take care to store them in such a way that the inside does not get soiled.

##### 3.2.2 Cleaning

- Soak the handles in a detergent solution.<sup>1</sup>
- Soak for 15 minutes to allow the solution to act, then clean by hand with a soft brush and a lint-free cloth.
- During cleaning, check regularly that the handles are fully clean and that no soiling remains on the inside or outside.

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- If any soiling remains, repeat cleaning or use an ultrasonic cleaning process.
- Rinsing: Rinse thoroughly in clean water to completely eliminate the detergent solution.
- Drying: Wipe with a clean lint-free cloth.

### 3.2.3 Disinfection

Handles may be disinfected by machine (GETINGE for example) and rinsed at a maximum temperature of 93°C. Typical recommended cycles:

Stage	Temperature	Time
Pre-washing	18 - 35° C	60 sec
Washing	46 - 50° C	300 sec
Neutralisation	41 - 43°C	30 sec
Washing 2	24 - 28°C	30 sec
Rinsing	92 - 93°C	600 sec
Drying		20 min

### 3.2.4 Sterilisation

After cleaning the handles must be steam sterilised as set out below:

Country	Sterilisation Cycle Type	Exp. Temp [°C]	Exp. Time [min]	Séchage [min]
USA & Canada	Prevacuum <sup>2</sup>	132 - 135	10	16
France	ATNC (Prion) (Prevacuum)	134	18	
Global	Prevacuum	Follow local/national regulations/requirements		

- Check that each handle is clean before continuing the process.
- Wrap the handles with sterilisation wrapper material (double wrapper or equivalent). Handles may also be placed in paper or plastic sterilisation bags<sup>3</sup> for easier identification and reuse.
- Place the handles on steriliser trays<sup>4</sup> with the opening downwards.
- Package with biological and/or chemical indicators for monitoring the sterilisation process. Follow applicable regulations.
- Run the sterilisation cycle according to the steriliser manufacturer's instructions.

 To ensure correct sterilisation do not allow any soiling to penetrate inside the handle.

 Handles are not guaranteed beyond 350 sterilisation cycles with the above sterilisation parameters.

- Dispose of the handles in the same way as other hazardous products in a hospital environment.

<sup>1</sup> Never soak the handles in enzyme-based detergents as they may damage the material used; rinse thoroughly if these detergents are used.

<sup>2</sup> This handle is made of a porous material.

<sup>3</sup> Possible suppliers of sterilization pouches:  
Medical Action Industries  
SBW Medical  
Baxter International

<sup>4</sup> For air removal and drying purposes.

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4.1 Recommended spare parts and consumables

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## 5 MAINTENANCE PROCEDURES

### 5.1 Disassembling a cover

- Remove the two CHc M4 attachment screws which hold the cover to the hinge.

#### Reassembly:

Perform the opposite operation.

-  Important: apply a drop of Loctite 243 on the screws during reassembly.

### 5.2 Disassembling a hinge

- Remove the two M5 nuts.
- Remove the two Ø 5 star washers which hold the hinge to the shell.
- Remove the hinge.
- Remove the hinge spacer beneath.

#### Reassembly:

Perform the opposite operation.

### 5.3 Disassembling the upper shell

- Open the covers.
- Unclip the springs from the upper shell and remove them.
  - 6 for HLX 3003
  - 6 for HLX 3004
  - 8 for HLX 3004TV
  - 8 for HLX 3005

-  Important: the upper and lower shells are connected by a ground continuity wire.

- Before removing the upper shell, remove the earth connection clip (located on the upper shell).

#### Reassembly:

Perform the opposite operation.

### 5.4 Disassembling the ergofocus

#### HLX3003:

- Remove the upper shell (see 5.3).
- Disconnect the 3 tie rods linking the ergofocus to the 3 reflectors by unclipping the 3 black plastic parts from the 3 swivel fittings on the ergofocus.
- Remove the plastic cam by withdrawing the two M4 screws from the centre of the ergofocus.
- Remove the washer.
- Remove the spring.
- Remove the washer.
- Remove the circlip.

-  Important: the ergofocus is held in place by the circlip and may fall after it is removed.

#### Reassembly:

Perform the opposite operation.

#### HLX3004:

- Remove the upper shell (see 5.3)
- Disconnect the tie rod linking the ergofocus to the central reflector by unclipping the black plastic part from the swivel fitting on the ergofocus.
- Remove the plastic cam by withdrawing the two M4 screws from the centre of the ergofocus.
- Remove the washer

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- e) Remove the spring
- f) Remove the washer
- g) Remove the circlip

 Important: the ergofocus is held in place by the circlip and may fall after it is removed.

- h) Remove the washer

**Reassembly:**

Perform the opposite operation.

**HLX3004TV - 3005:**

- a) Remove the upper shell (see 5.3)
- b) Disconnect the tie rod linking the ergofocus to the central reflector by unclipping the black plastic part from the swivel fitting on the ergofocus.
- c) Remove the metal cam by withdrawing the two M4 screws from the centre of the ergofocus.
- d) Remove the washer
- e) Remove the spring
- f) Remove the washer
- g) Remove the circlip

 Important: the ergofocus is held in place by the circlip and may fall after it is removed.

- h) Remove the washer

**Reassembly:**

Perform the opposite operation.

### 5.5 Disassembling the side grab handles

- a) Remove the upper shell (see 5.3)
- b) Remove the Ejoy screw
- c) Remove the Belleville washer
- d) Remove the star washer
- e) Remove the side handle

### 5.6 Disassembling a lens

- a) Hold the lens in place with a hand from below
- b) Turn the three lens retaining screws by one quarter-turn

**Reassembly:**

- a) Check that the seal is correctly located around the lens
- b) Tighten the quarter-turn screws to hold the lens in place

### 5.7 Disassembling a reflector

- a) Remove the lens (see 5.6)
- b) Lock the M4 nut and unscrew the M4x5 screw
- c) Repeat with the second shell/reflector mounting point
- d) Retrieve the reflector, the two screws, two nuts and two star washers

**Reassembly:**

- a) Grease the reflector mounting inserts with Loctite 8105
- b) Attach the reflector to the shell using the M4x5 screws
- c) Position two star washers on the screw
- d) Tighten the M4 nuts on the two M4x5 screws

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### 5.8 Disassembling a bulb holder

- Disconnect the two bulb holder power cables from the terminal block
- Remove the two bulb holder mounting lugs (sides)
- Remove the bulb holder

#### Reassembly:

- Connect the two leads from the bulb holder to the terminal block
- Run the cable (see figure 1)
- Use a cable tie to attach the cables together again

### 5.9 Adjustment of the mounting mechanism of the camera

If the dummy camera or camera cannot be mounted, is too hard to clip in place or if the gap with the shell is too large, the mounting mechanism must be adjusted.

#### Figure

- Use a 7 mm tubular wrench to loosen the three H M4x16 bolts (2). Caution: do not unscrew the bolts completely.
- The bayonet (1) can then be adjusted in direction "a" (increased clearance, more flexible mounting) or direction "b" (reduced clearance, firmer mounting).
- Use a 7 mm tubular wrench to re-tighten the three H M4x16 bolts (2). Caution: the bayonet (1) must not have any play.
- Try mechanism several times with the dummy camera AND the real camera, adjusting again if necessary.

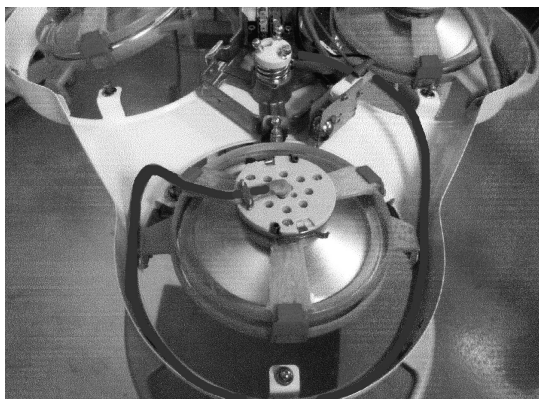
Fig. 1

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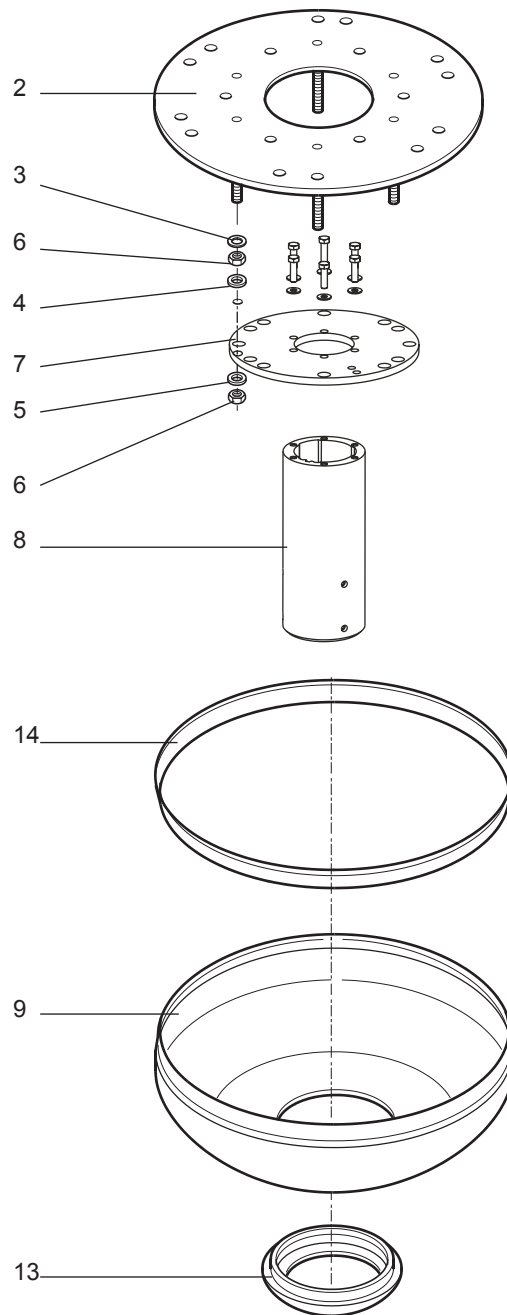
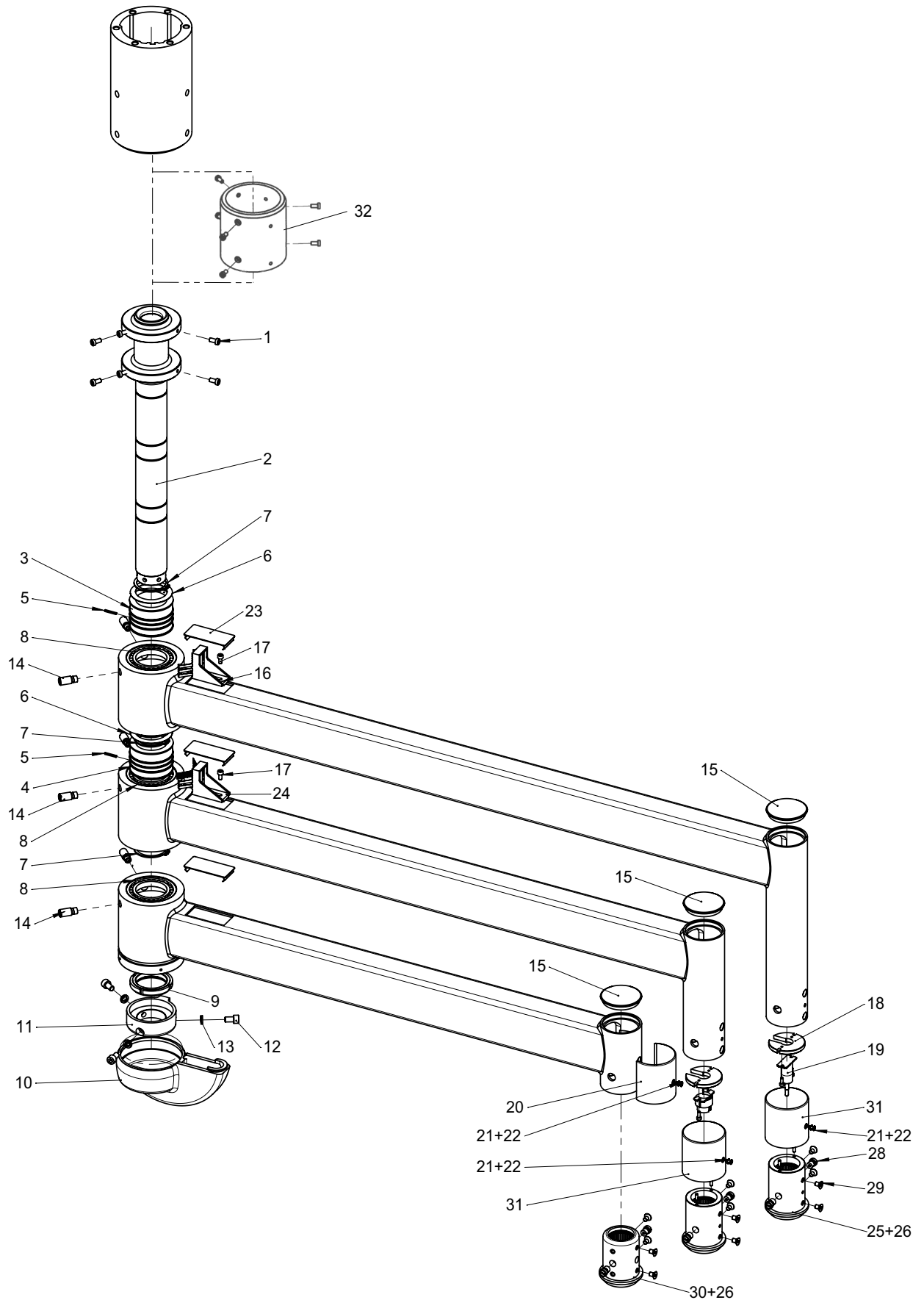
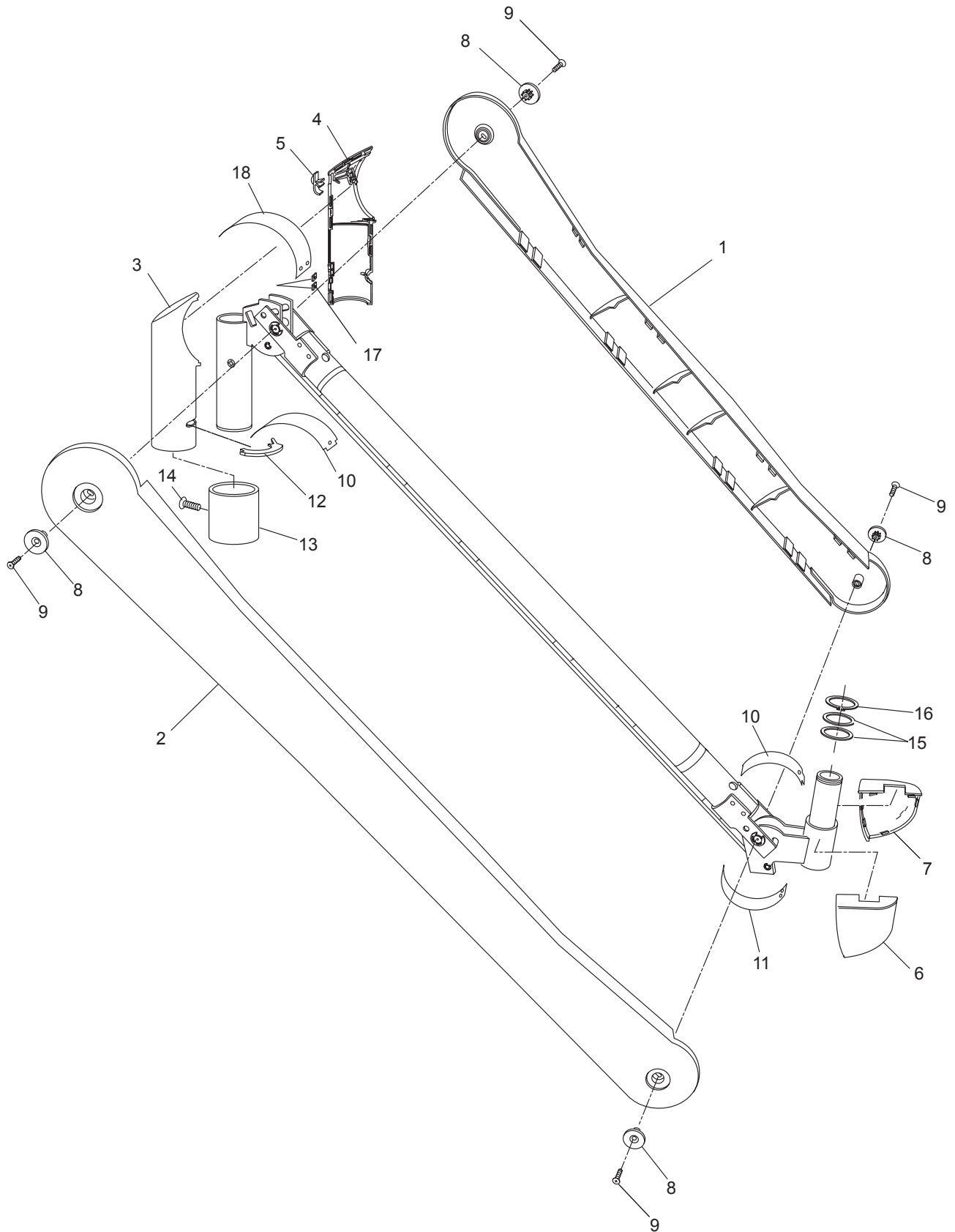
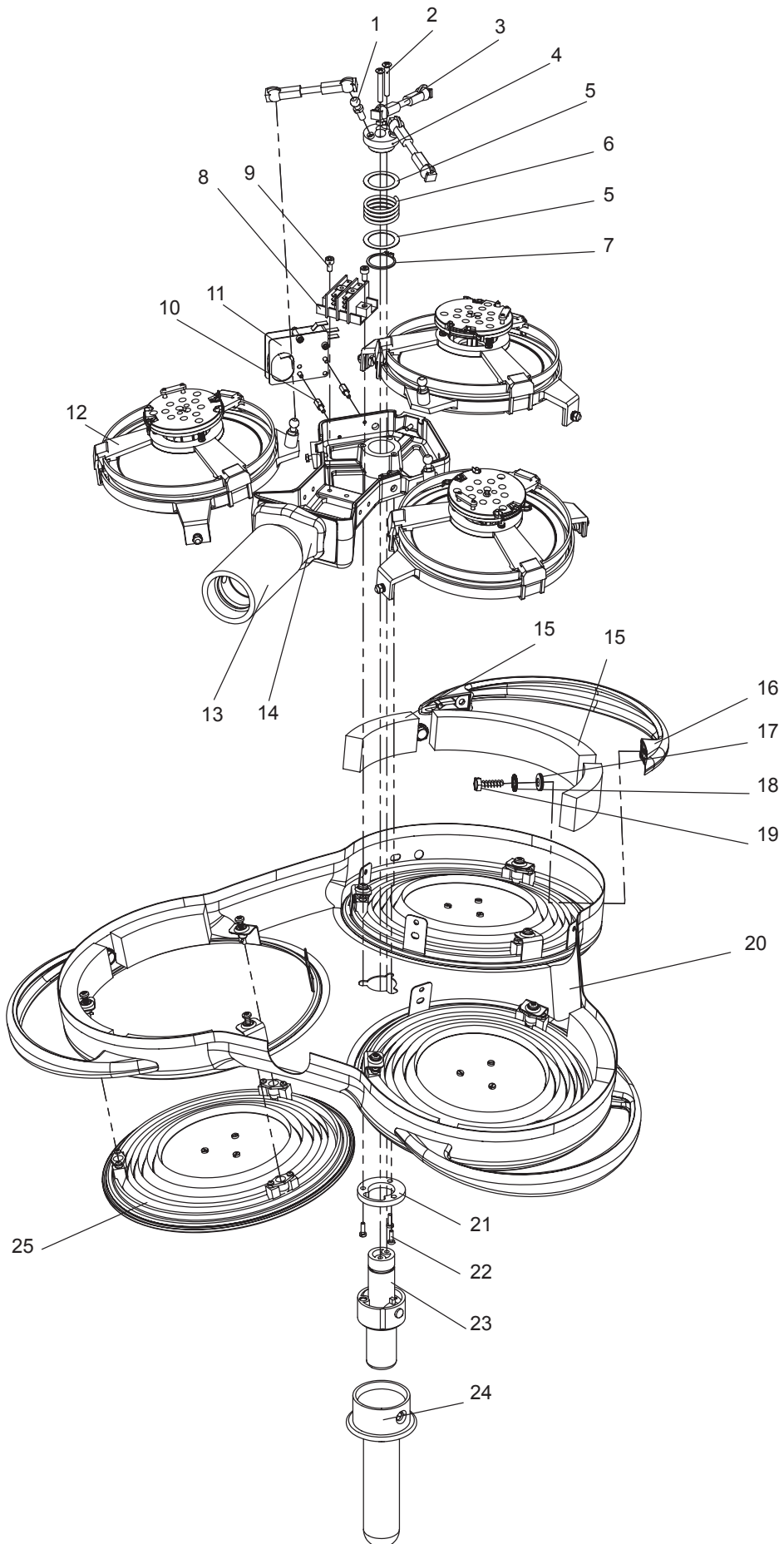
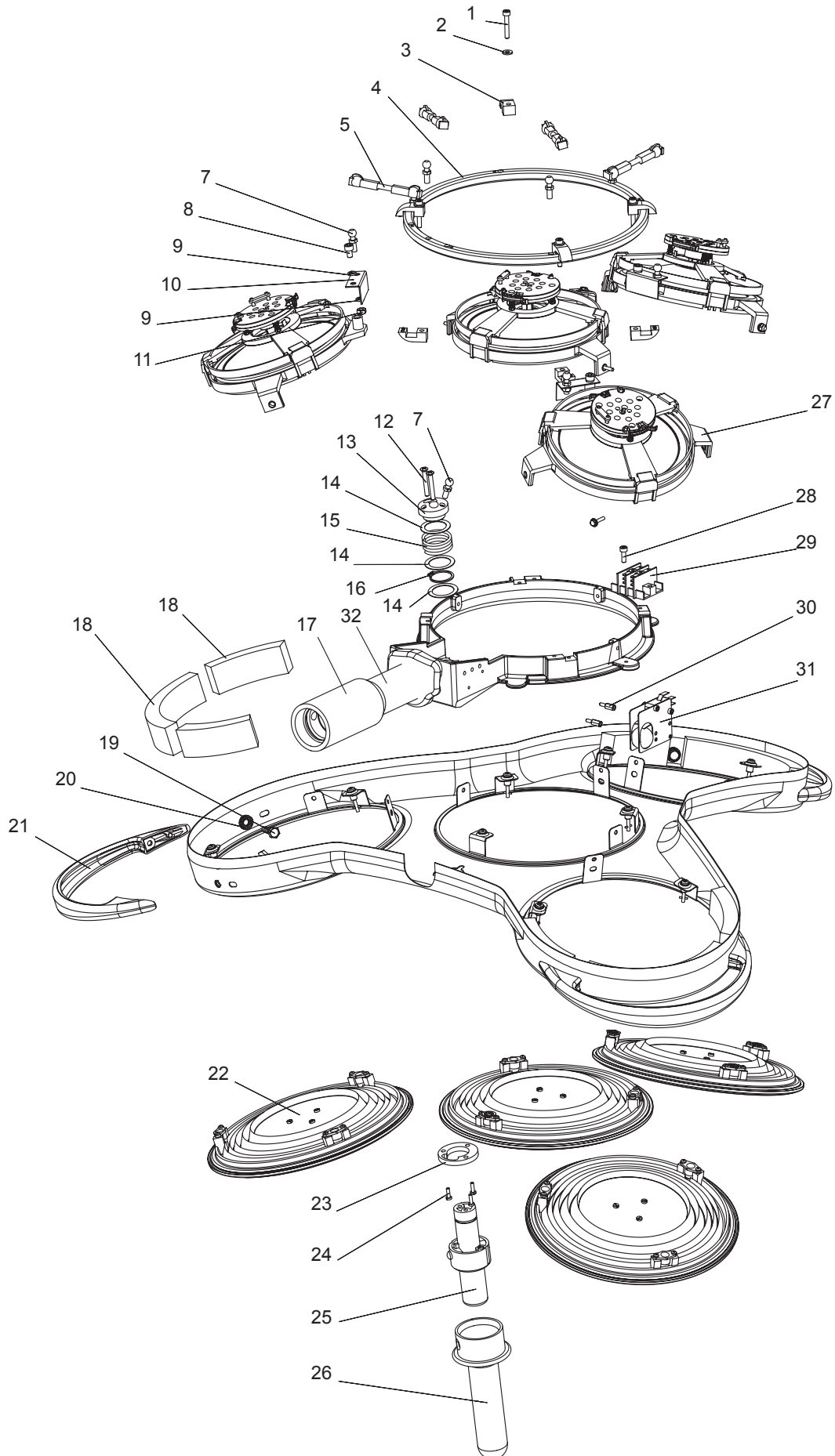
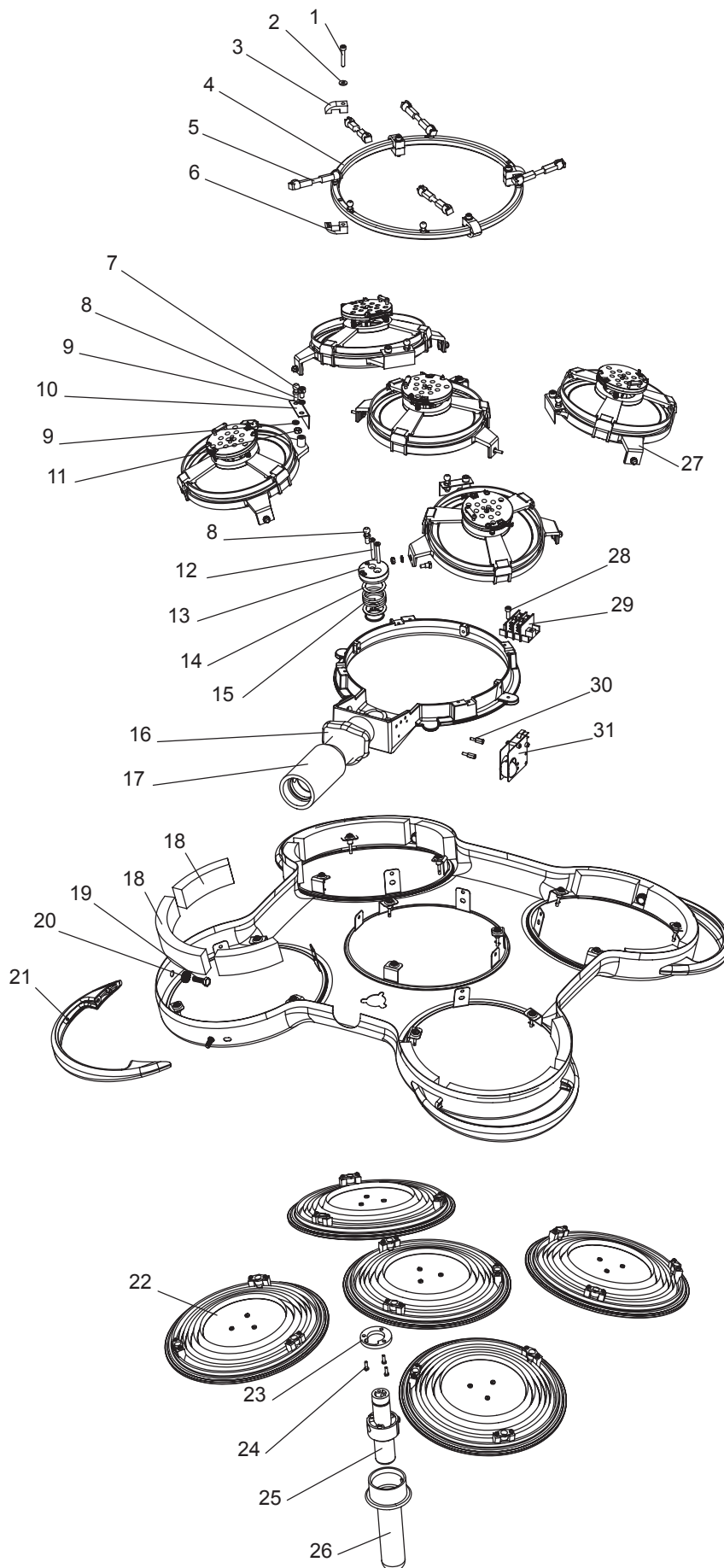
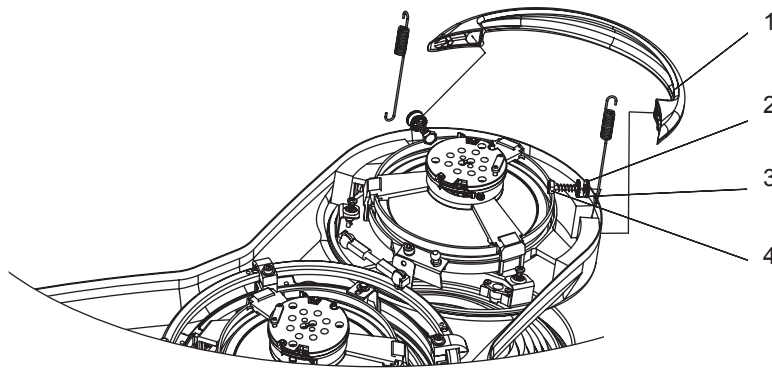
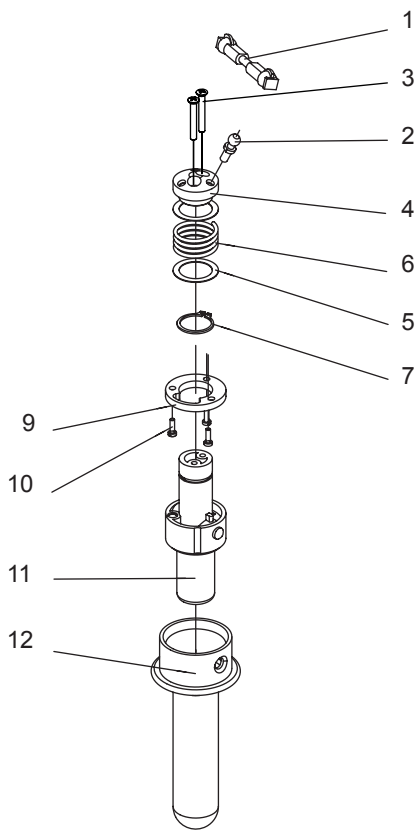
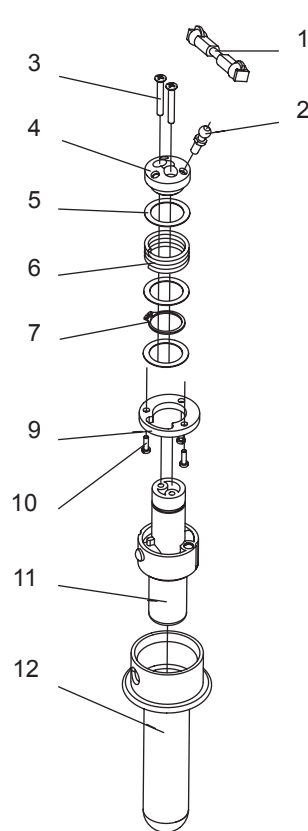
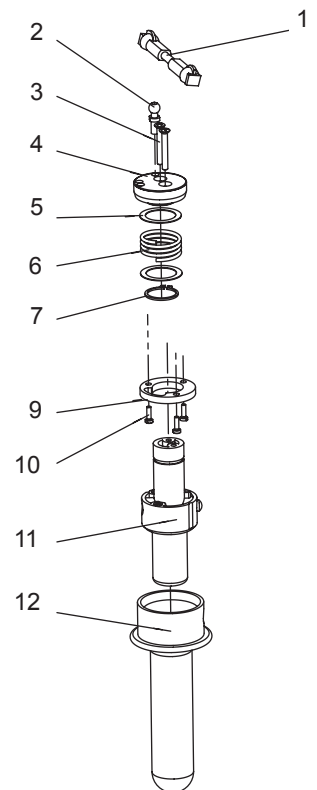
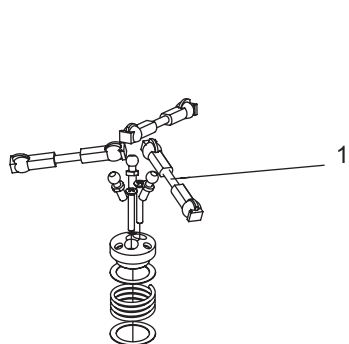
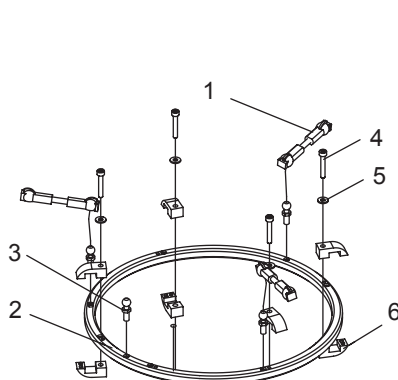
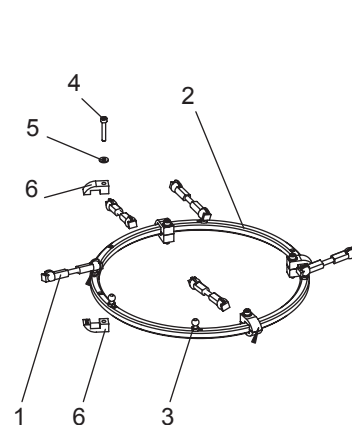
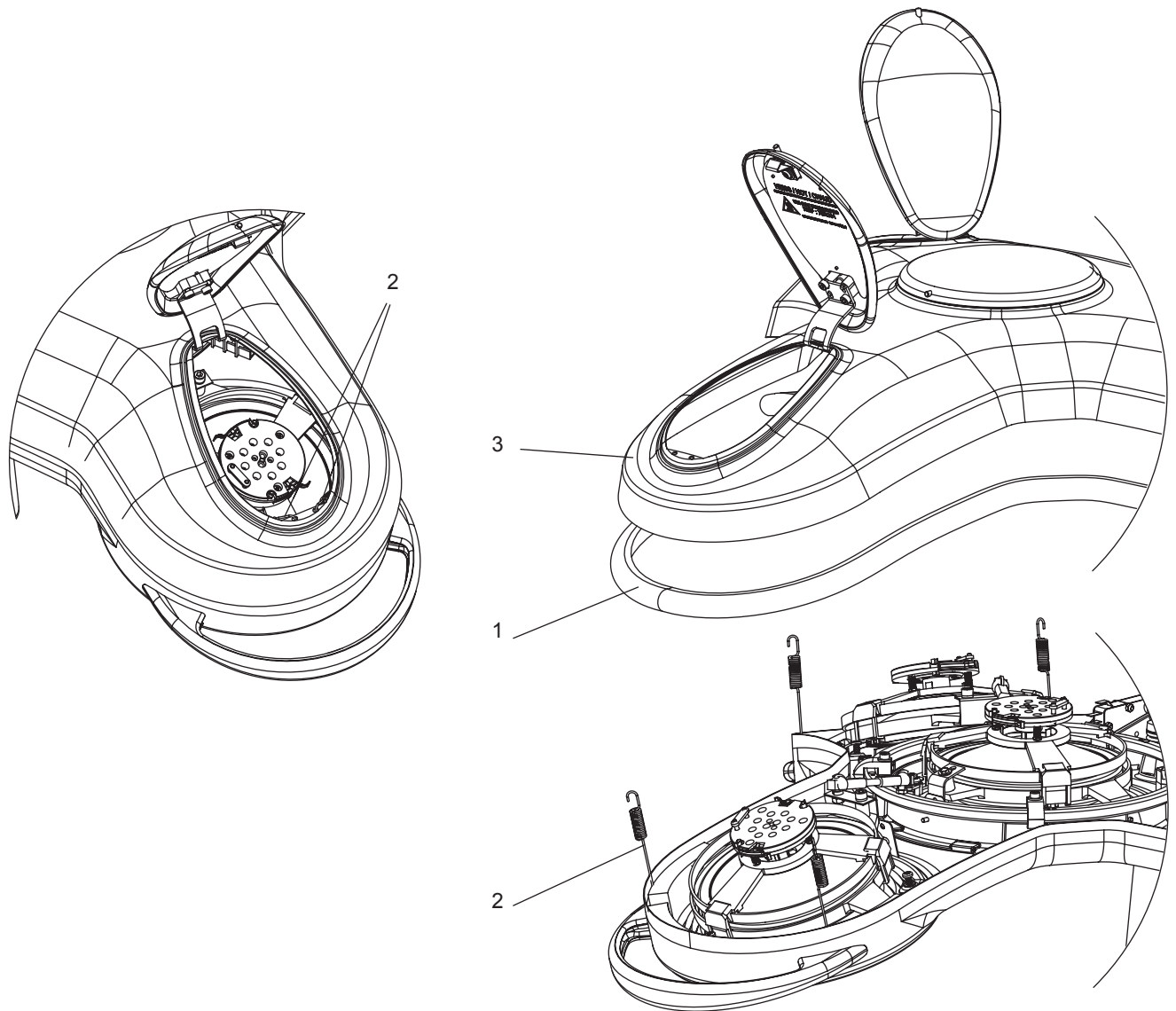
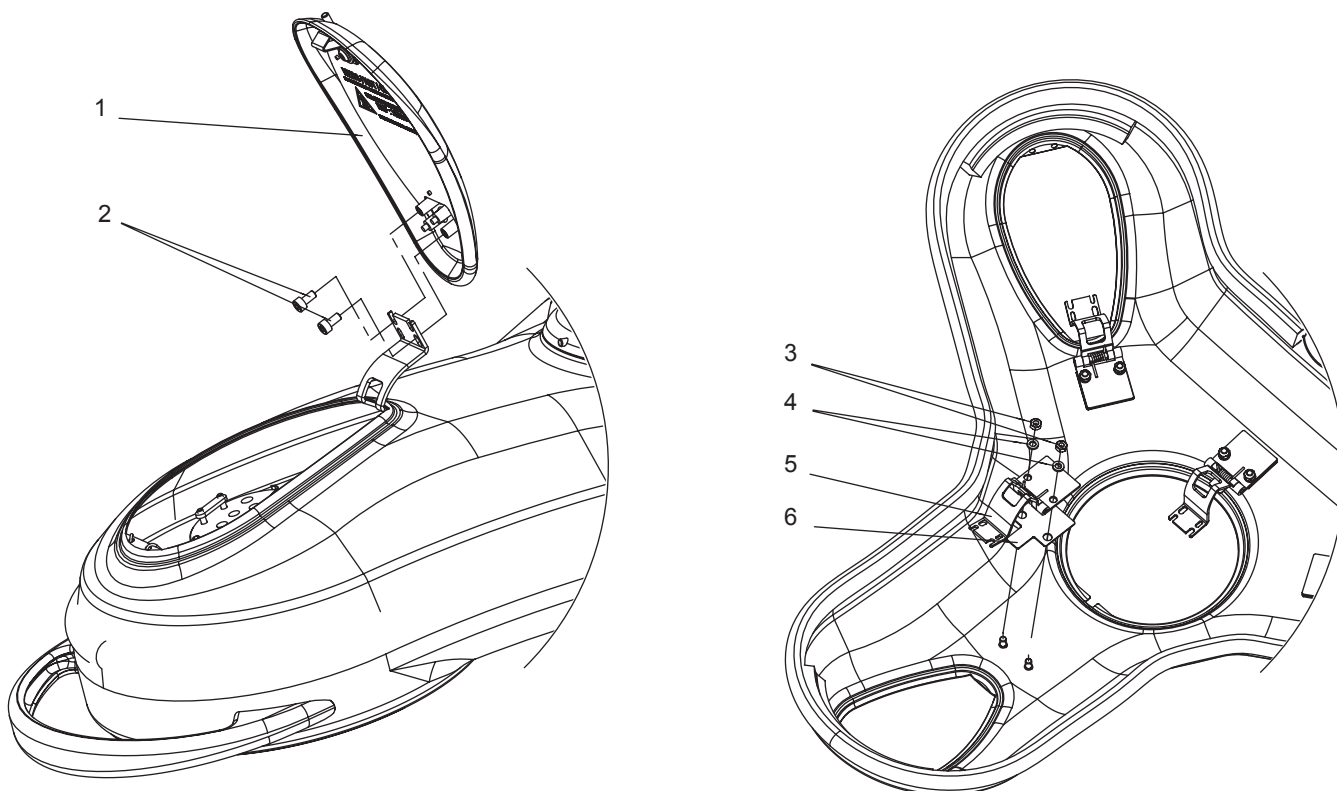
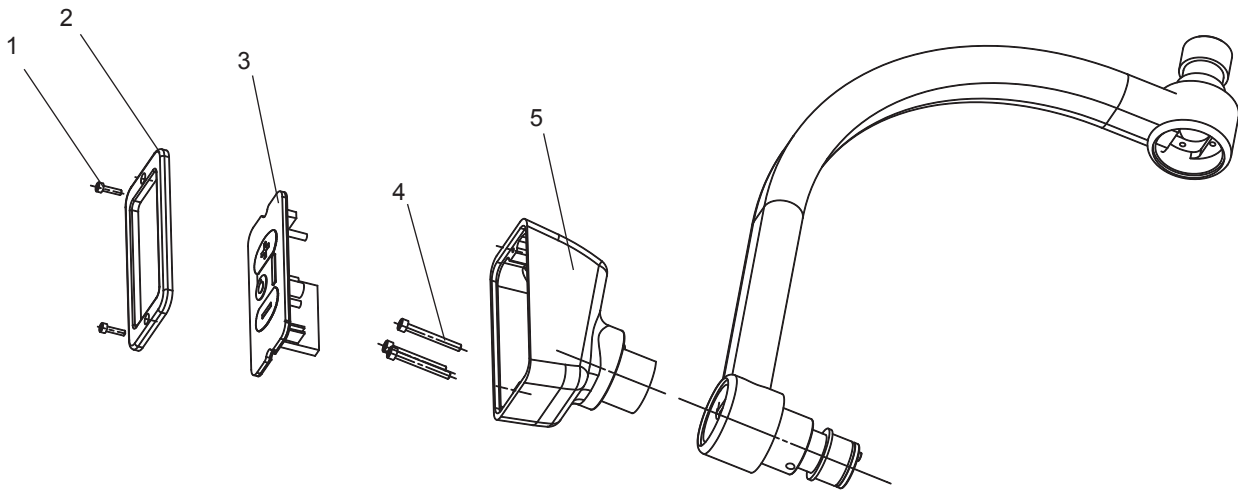
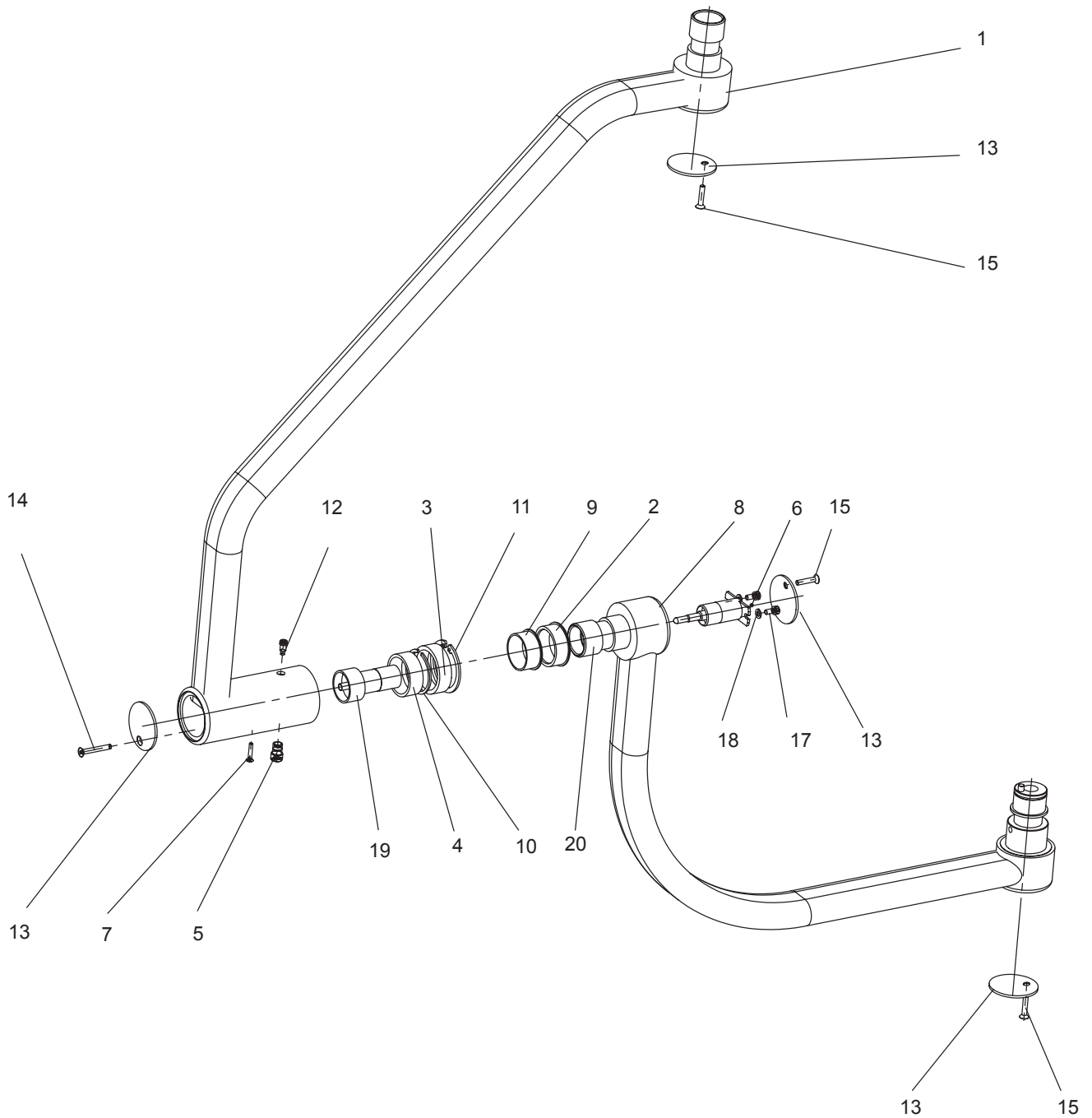
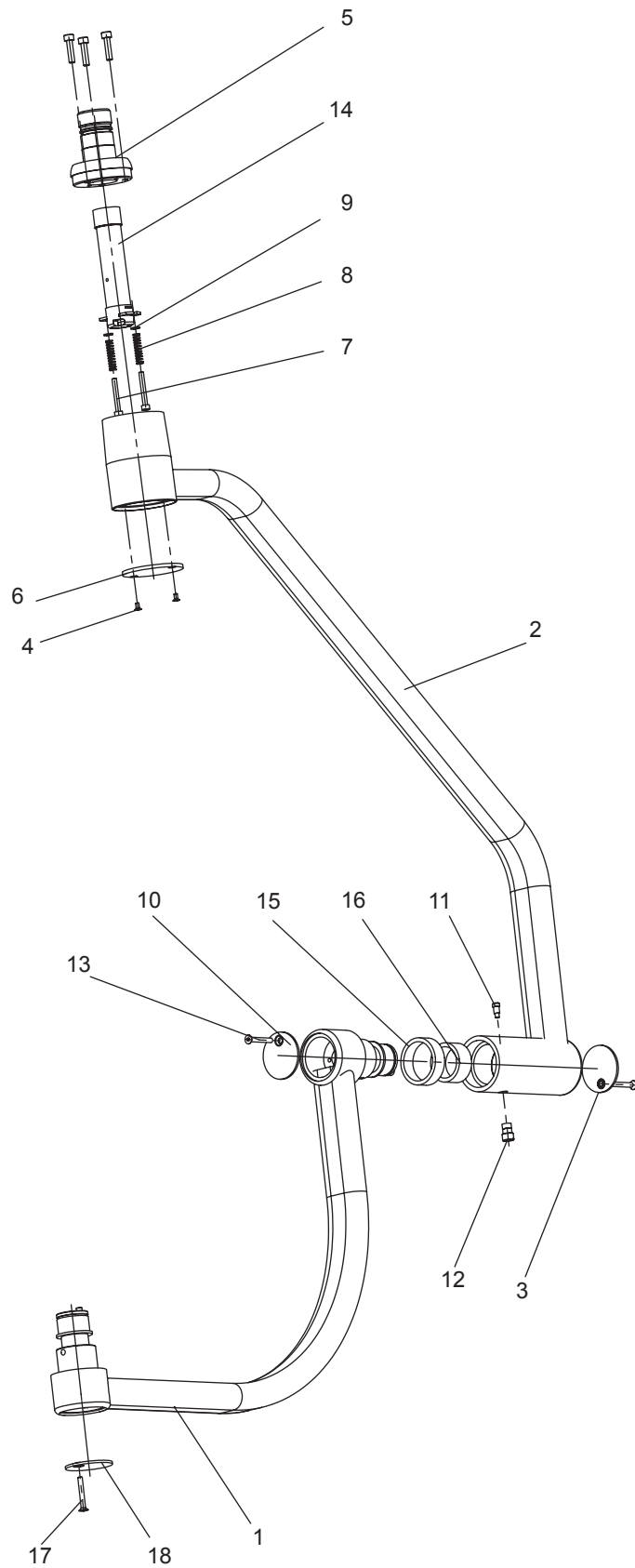
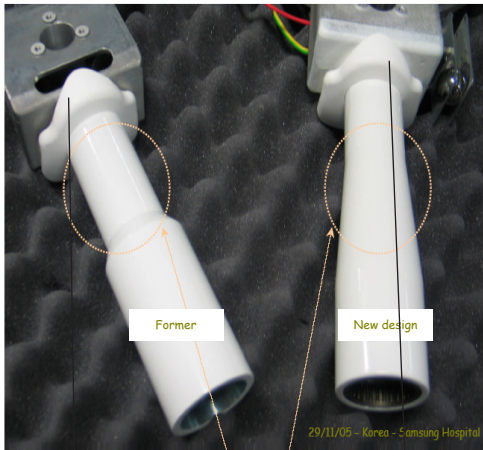
7 FORKS

7.1 Oblong forks

For balance reasons, there are two types of oblong fork which must be assembled with their respective racket pieces.

First-generation racket piece

Latest-generation racket piece

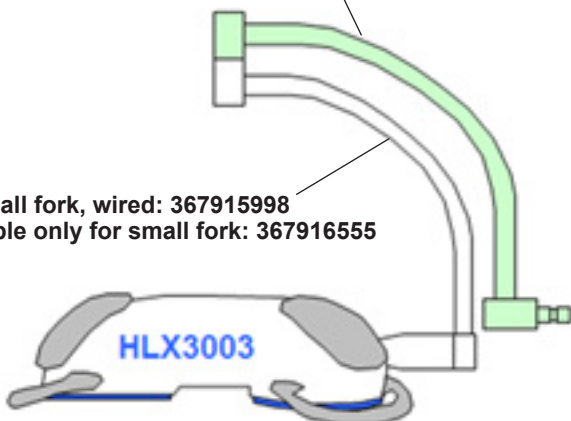
The two types of racket piece are easily differentiated by their shape (see diagrams above). The oblong forks fitted on first-generation racket pieces are no longer available and must be replaced with latest-generation round forks.

Oblong forks fitted on latest-generation racket pieces are available while stocks last. Where necessary, they must be replaced with round forks.

OVERVIEW OF OBLONG FORKS (fitted on latest-generation racket pieces)

Comfort fork, wired: 567906999  
Cables only for both forks: 367918555

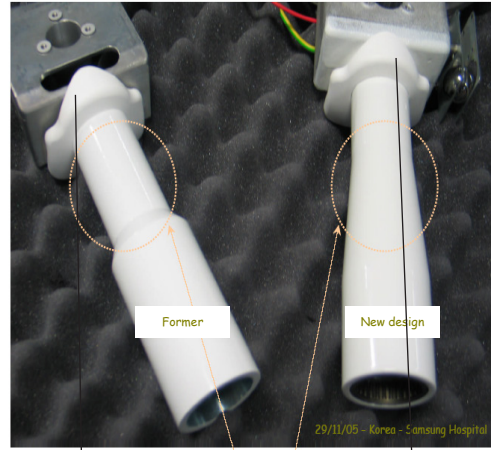
Small fork, wired: 367915998  
Cable only for small fork: 367916555

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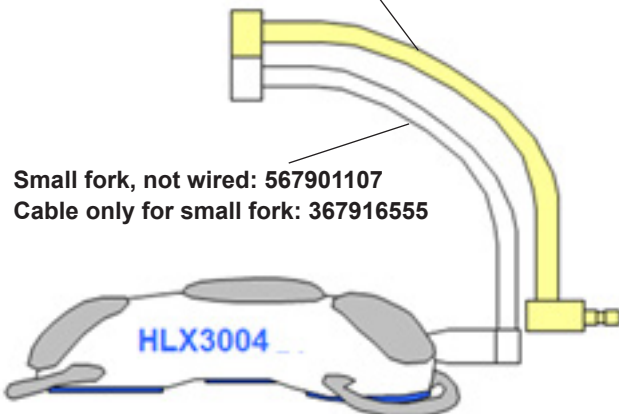
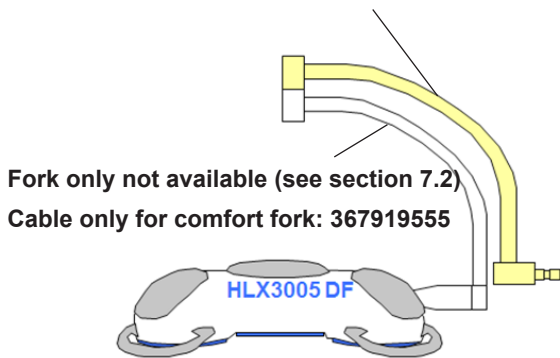
Comfort fork, wired: 567907999  
Cables only for both forks: 367918555

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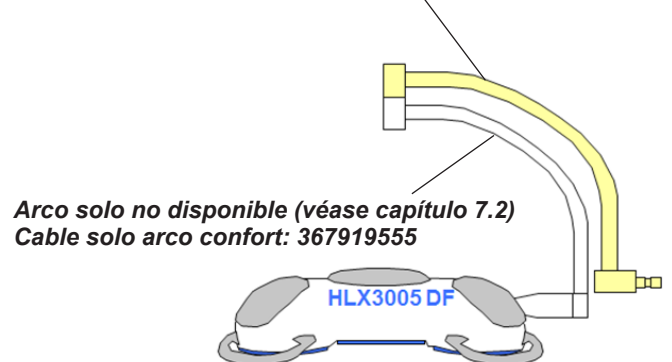
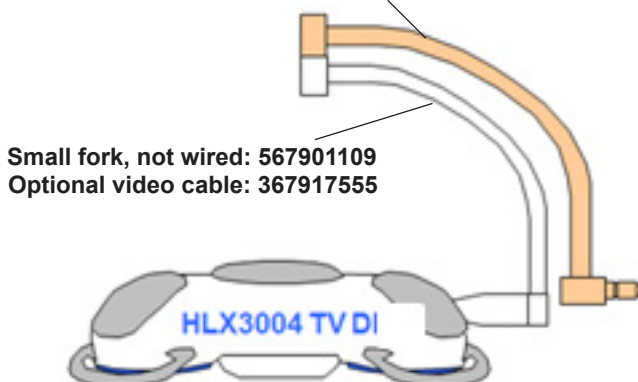
Comfort fork only not available (see section 7.2)  
Cable only for small fork (367916555)

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**Information:** the stop pin used to join the fork to the lighthead is not supplied with the forks (pin kit article code 367935555).

For single fork configurations, select the product codes for small forks (e.g. for HLX3003SF ARD367915998).

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## 7.2 Round forks (new generation)

Round forks supersede oblong forks.

For HLX3004 and 3005 lighthoods only, the round forks are not compatible with first-generation racket pieces. Whenever oblong forks are replaced by round forks, the first-generation racket pieces must also be replaced (see product codes below).

HLX3003 lighthoods with first-generation racket pieces are compatible with round forks.

The replacement procedures are described on pages 13 to 15 of the HLX3000 technical manual. MAQUET SAS can also replace the racket pieces in its workshops (please enter a price request in Magic for this).

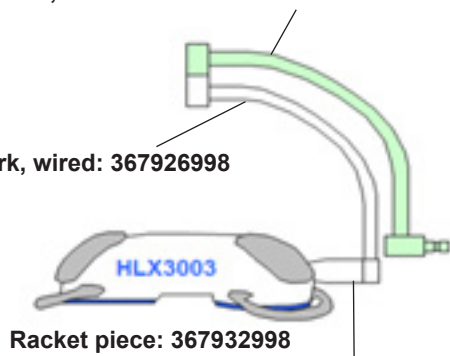
### OVERVIEW OF ROUND FORKS (fitted on latest-generation racket pieces)

**Information:** the stop pin used to join the fork to the lighthood is not supplied with the forks (pin kit article code 367935555).

For single fork configurations, select the product codes for small forks (e.g. for HLX3003SF ARD367926998).

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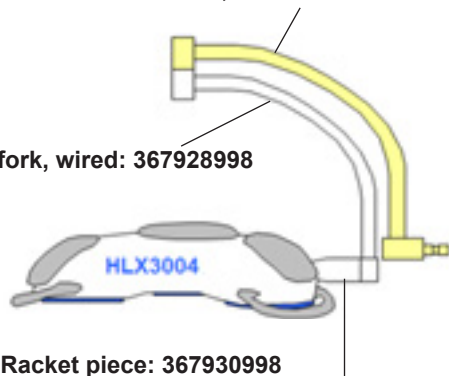
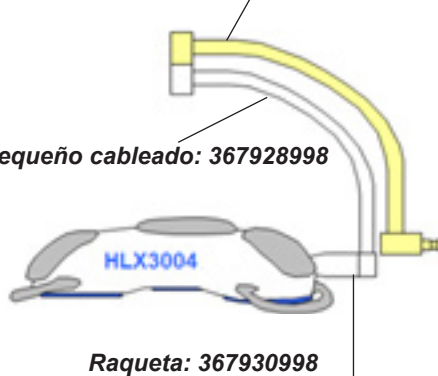
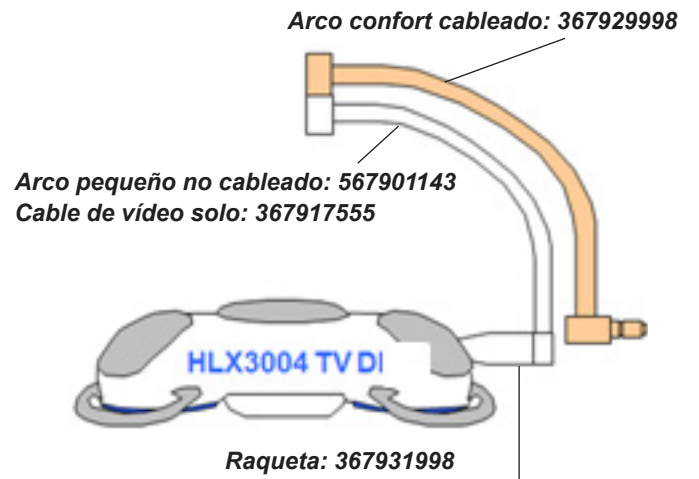
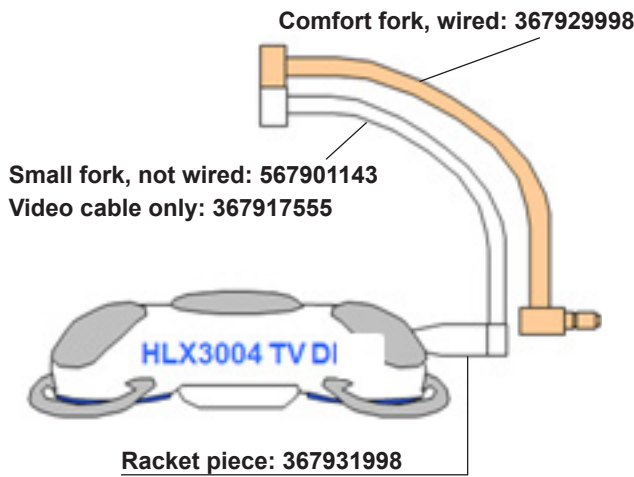
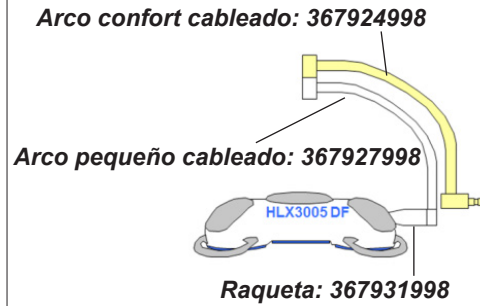
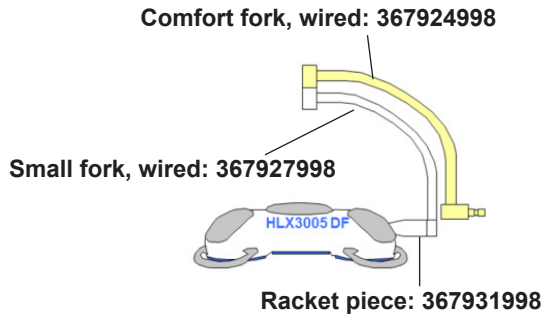
Arco confort cableado: 367924998

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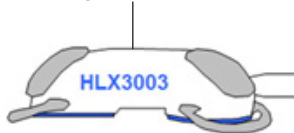
## 8 SPRING ARMS

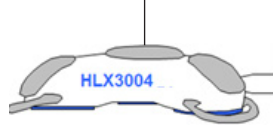
Spring arms are compatible with both types of fork (round and oblong).

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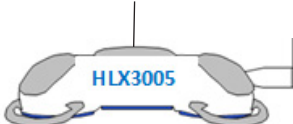
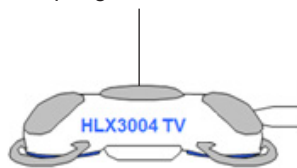
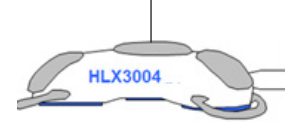
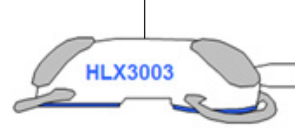
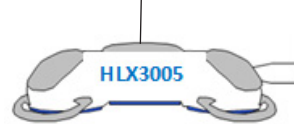
## 9 TROUBLESHOOTING

ANOMALY	PROBABLE CAUSE	CORRECTIVE ACTION
• A cupola does not light up.	• Bulb failure or missing bulb.	• Switch off power supply and replace bulb(s).
	• The cupola is turned upwards.	• Turn the cupola with light directed down.
	• Mains power failure.	• Check if another piece of equipment is operating on the same electrical power system.
	• Other cause.	• Contact a certified MAQUET technician.
• All cupolas are not lit up.	• Each cupola has its own individual control.	• Check that all power supplies are properly switched on (green indicator).
• Low bulb service life.	• Improper bulb or overvoltage condition.	• Make sure that you are using the required MAQUET bulbs exclusively. Do not use bulbs used with surgical lights of a previous generation (HANAULUX 2000), which are not compatible in power and size.
		Check voltage at the bulb pins. $22,8V_{EFF} (V_{AC+DC}^{RMS})$
• Sterilizable handle does not fit into its support.	• Non application of sterilization requirements (temperature, time).	• Check operation of handle lock mechanism (click) and overall handle.
	• Maximum service life exceeded / Handle is out of shape.	• Replace handle.
• Cupola drifts.	• Suspension tube is not levelled.	• Check level and ceiling structure.
	• Unstable ceiling structure.	• Contact a certified MAQUET technician.
	• Brakes are not sufficiently tightened.	• Adjust the brake.
• Cupola too loose or too difficult to handle.	• Brake misadjustment.	• Check balance adjustment.
	• Lack of lubrication.	• Contact a certified MAQUET technician.
• The dummy or camera cannot be mounted	• Bayonet fitting not correctly adjusted	• Contact a certified MAQUET technician.
• Significant gap between dummy or camera and cupola	• Bayonet fitting not correctly adjusted	• Contact a certified MAQUET technician.

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SERVICE PROTOCOL	
For use during Installation / Servicing	
<h1 style="margin: 0;">HLX 3000</h1>	<input type="checkbox"/> Contract    Order n°..... <input type="checkbox"/> Chargeable <input type="checkbox"/> Other:.....
<input type="checkbox"/> <b>Installation</b> Delivery date: ..... Installation date: ..... Commissioning date: .....	
<input type="checkbox"/> <b>Servicing</b> Preventive / Curative                      Valid until: .....	
Hospital:	
Building: .....	Floor level: .....
Service: .....	Operating room: .....
Product(s) identification	
Cupola n°1	Ref. number: ..... S/N: .....
Cupola n°2	Ref. number: ..... S/N: .....
Cupola n°3	Ref. number: ..... S/N: .....
Overall assessment	
<input type="checkbox"/> Configuration fully operational	<input type="checkbox"/> Configuration shall not be used until all deficiencies are corrected
<input type="checkbox"/> Free of direct risk but deficiencies detected. May be corrected on short term.	<input type="checkbox"/> Configuration no longer safe. Taking out of service is recommended.
Remarks	
..... ..... ..... .....	
Signature	
Name: .....  Date: .....	Name: .....  Function: .....
Installer/Technician authorized by MAQUET	Customer

Measuring equipment used

IEC 62353 compatible control units

Type: **Voltmeter**

S/N: .....

Calibration valid until: .....

Type: **Luxmeter**

S/N: .....

Calibration valid until: .....

Type: **Megohmmeter**

S/N: .....

Calibration valid until: .....

Type: .....

S/N: .....

Calibration valid until: .....

Electrical safety test

The protective earth resistance is to be measured between point **0** and the point **3**

Protective earth resistance  $\leq$  300 m $\Omega$

Measured: ..... m $\Omega$

Pass

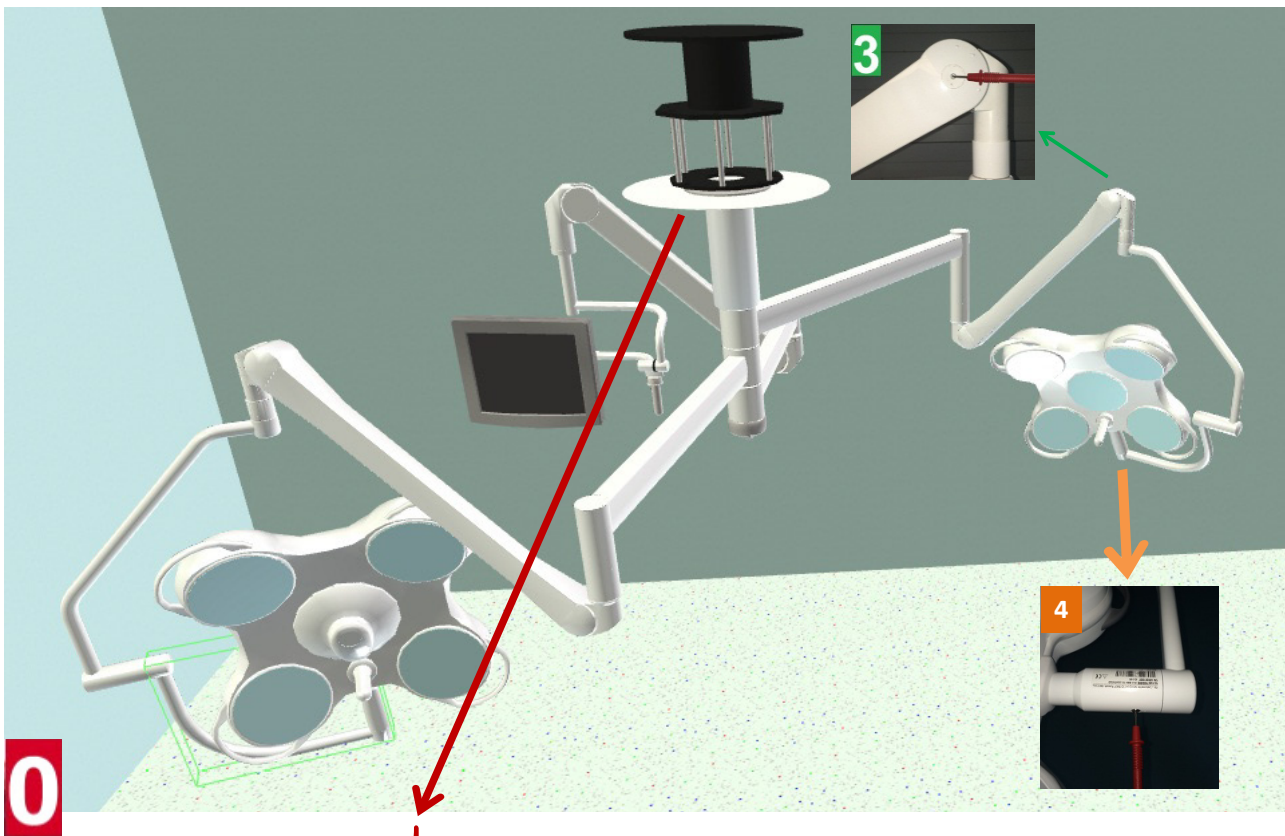
Fail

The continuity test is to be performed between point **3** and the point **4**

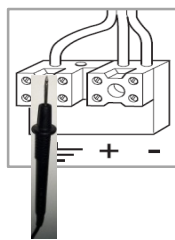
Continuity test

Pass

Fail

MEASURING POINT DEPENDING OF THE INSTALLATION

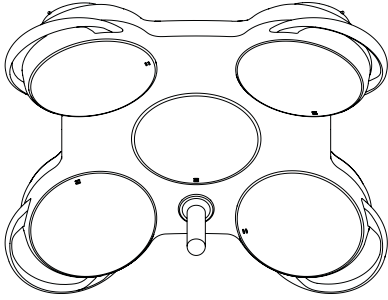
Ensure proper calibration of your measuring equipment before EACH reading.

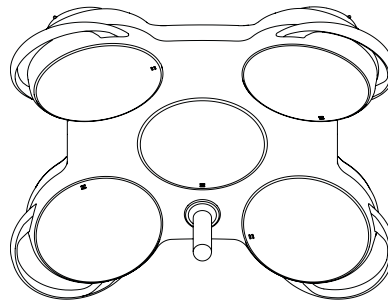
Please remove the identification sticker from the packaging and place it in one of the below window frame.

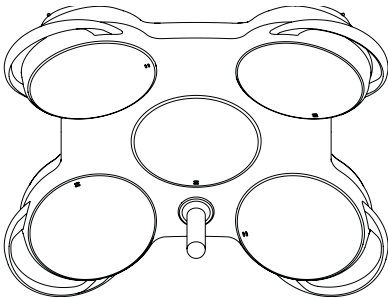
Cupola 1 with S/N

Cupola 2 with S/N

Cupola 3 with S/N

**Note:** if applicable, please staple the delivery note with this document.

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Mesuré : ..... lux

## Setting

- Balance of the spring arm
- Vertical stop of the spring arm

**⚠ A spring arm wears out in time and should be replaced every 6 years**

Replaced?

Yes      No

- Test the equipment is easy to handle

## Checking

- Hardness of the suspension by shaking the configuration
- Verticality of the suspension tube
- 6 fixing screws pre-glued (refer to the appendix in the Technical Manual)
- Fixing of the plastic covers on the spring arm
- Fixing and proper movement of the shutters
- Safety segment is positioned correctly (dismantle and lubricate if necessary)

**⚠ A safety segment wears out in time and should be replaced every 2 years. Replaced ?**

- Safety sleeve present and positioned with its fixing screw
- Assembly of the joints for each arm. Tightening of the screw and positioning of the retaining ring

Yes      No

- N/A  Sterilizable handle holder

- Sterilizable handle engages correctly and stays in place

- N/A  Fixing of the outer handles

- Tight fitting of the wire into the metal connector plug (the one connected to the kit detector)

- Limit stop on the yoke for the cupola rotation is functional

- N/A  Keypad operates correctly

- System switches to battery mode and the ON/OFF button LED on the keypad turns orange and switches back to mains

- Opening and closing of the lamp cover

- Replace all the bulbs

- N/A  Light patch, adjusting the articulated rods if necessary

- Functioning of the emitter / receiver system

- Camera operates correctly (functions and image)

- Hanauvision box operates correctly

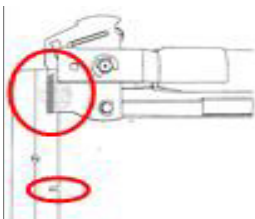
- WPS is connected to the ground

- All boards are properly fixed inside the WPS

- N/A  The correct setting on the regulator boards respective to the cupola

- Software version in START UP menu. If inferior to V5.40 → obsolete, inform the customer

- Front pivot on Acrobat 2000 spring arm (see NIT 205 for more details)

**Change spring arm if any sign of crack is seen.**

Replaced ?

Yes      No

## Tightening

- Collar around the power supply connector at the ceiling tube
- Wires on the power supply connector at the ceiling tube
- Ceiling covers + proper positioning
- Wires on the connectors' terminal heads inside the WPS
- Wires on the battery pack
- All visible screws
- Presence of the stopper on the concerned suspension

## Appearance

- All seals hold correctly and are not worn
- General appearance of the underside (no scratches, no cracks)
- No corrosion anywhere
- No paint chip anywhere
- Appearance of the keypad
- Attachment of the WPS on the wall
- Appearance of the WPS cover
- Holding of the cover and the hinges of the WPS
- Cleaning the complete configuration
- Check the label bearing the CE marking

***We do not recommend the use of alcoholized solution***

SERVICE PROTOCOL	
For use during Installation / Servicing	
<h1 style="margin: 0;">HLX 3000</h1>	<input type="checkbox"/> Contract    Order n°..... <input type="checkbox"/> Chargeable <input type="checkbox"/> Other:.....
<input type="checkbox"/> <b>Installation</b> Delivery date: ..... Installation date: ..... Commissioning date: .....	
<input type="checkbox"/> <b>Servicing</b> Preventive / Curative                      Valid until: .....	
Hospital:	
Building: .....	Floor level: .....
Service: .....	Operating room: .....
Product(s) identification	
Cupola n°1	Ref. number: ..... S/N: .....
Cupola n°2	Ref. number: ..... S/N: .....
Cupola n°3	Ref. number: ..... S/N: .....
Overall assessment	
<input checked="" type="checkbox"/> Configuration fully operational	<input type="checkbox"/> Configuration shall not be used until all deficiencies are corrected
<input type="checkbox"/> Free of direct risk but deficiencies detected. May be corrected on short term.	<input type="checkbox"/> Configuration no longer safe. Taking out of service is recommended.
Remarks	
..... ..... ..... .....	
Signature	
Name: .....  Date: .....	Name: .....  Function: .....
Installer/Technician authorized by MAQUET	Customer

### Measuring equipment used

IEC 62353 compatible control units

Type: **Voltmeter**

S/N: .....

Calibration valid until: .....

Type: **Luxmeter**

S/N: .....

Calibration valid until: .....

Type: **Megohmmeter**

S/N: .....

Calibration valid until: .....

Type: .....

S/N: .....

Calibration valid until: .....

### Electrical safety test

The protective earth resistance is to be measured between point **0** and the point **3**

Pass

Fail

Protective earth resistance  $\leq$  300 m $\Omega$

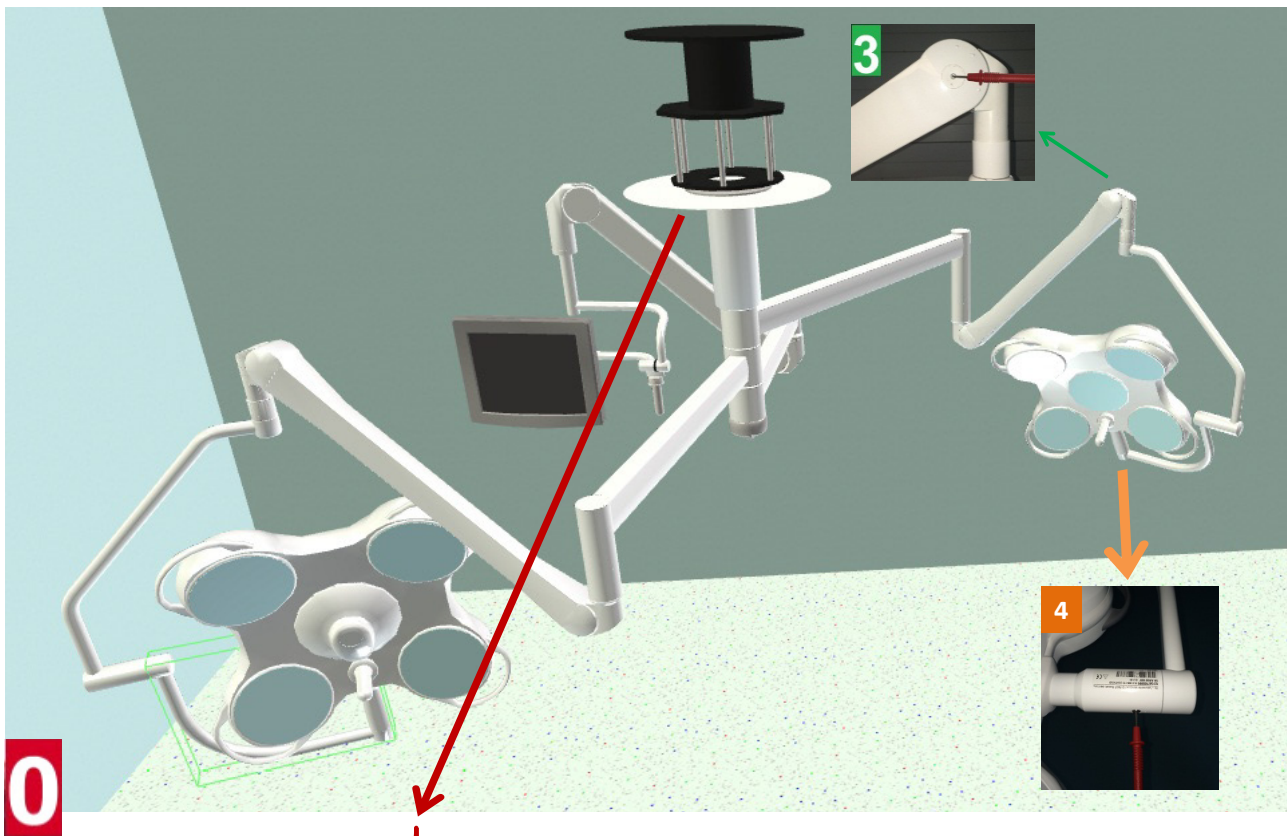
Measured: ..... m $\Omega$

The continuity test is to be performed between point **3** and the point **4**

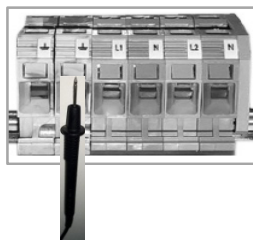
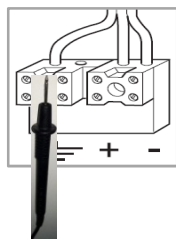
Pass

Fail

Continuity test

#### MEASURING POINT DEPENDING OF THE INSTALLATION

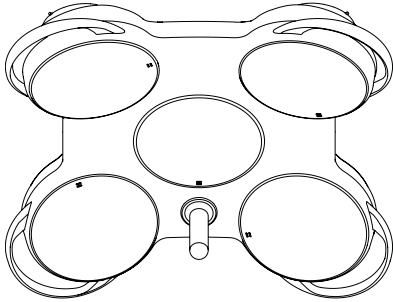
Ensure proper calibration of your measuring equipment before **EACH** reading.

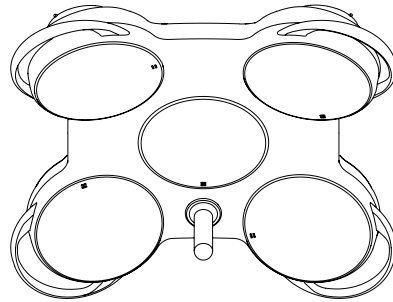
Please remove the identification sticker from the packaging and place it in one of the below window frame.

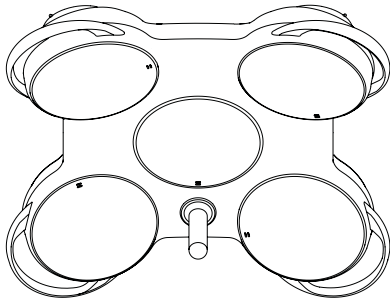
Cupola 1 with S/N

Cupola 2 with S/N

Cupola 3 with S/N

**Note:** if applicable, please staple the delivery note with this document.

## Maintenance

- Replacement of : ..... S/N : .....
- Replacement of : ..... S/N : .....
- Replacement of : ..... S/N : .....
- MAQUET batteries replaced

## Remarks

## Illumination verification

→ Minimum value : 40 000 lux, maximum value allowed : 160 000 lux

- Measure the bulb voltage.  $U = 22,8 \pm 1V$  (AC + DC) U = .....V

HLX 3003 → 80 000 lux

HLX 3004 → 110 000 lux

HLX 3005 → 140 000 lux

HLX 3004 TV → 100 000 lux

HLX 300..... : S/N : .....

Mesuré : ..... lux

HLX 300..... : S/N : .....

Mesuré : ..... lux

HLX 300..... : S/N : .....

Mesuré : ..... lux

## Setting

- Balance of the spring arm
- Vertical stop of the spring arm

**⚠ A spring arm wears out in time and should be replaced every 6 years**

Replaced?    
Yes No

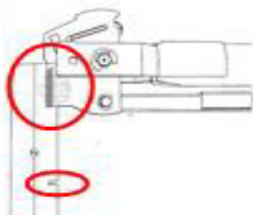
- Test the equipment is easy to handle

## Checking

- Hardness of the suspension by shaking the configuration
- Verticality of the suspension tube
- 6 fixing screws pre-glued (refer to the appendix in the Technical Manual)
- Fixing of the plastic covers on the spring arm
- Fixing and proper movement of the shutters
- Safety segment is positioned correctly (dismantle and lubricate if necessary)

**⚠ A safety segment wears out in time and should be replaced every 2 years. Replaced ?**

- Safety sleeve present and positioned with its fixing screw Yes  No
- Assembly of the joints for each arm. Tightening of the screw and positioning of the retaining ring
- Sterilizable handle holder
- Sterilizable handle engages correctly and stays in place
- Fixing of the outer handles
- Tight fitting of the wire into the metal connector plug (the one connected to the kit detector)
- Limit stop on the yoke for the cupola rotation is functional
- Keypad operates correctly
- System switches to battery mode and the ON/OFF button LED on the keypad turns orange and switches back to mains
- Opening and closing of the lamp cover
- Replace all the bulbs
- Light patch, adjusting the articulated rods if necessary
- Functioning of the emitter / receiver system
- Camera operates correctly (functions and image)
- Hanauvision box operates correctly
- WPS is connected to the ground
- All boards are properly fixed inside the WPS
- The correct setting on the regulator boards respective to the cupola
- Software version in START UP menu. If inferior to V5.40 → obsolete, inform the customer
- Front pivot on Acrobat 2000 spring arm (see NIT 205 for more details)

**Change spring arm if any sign of crack is seen.**

Replaced ?    
Yes No

## Tightening

- Collar around the power supply connector at the ceiling tube
- Wires on the power supply connector at the ceiling tube
- Ceiling covers + proper positioning
- Wires on the connectors' terminal heads inside the WPS
- Wires on the battery pack
- All visible screws
- Presence of the stopper on the concerned suspension

## Appearance

- All seals hold correctly and are not worn
- General appearance of the underside (no scratches, no cracks)
- No corrosion anywhere
- No paint chip anywhere
- Appearance of the keypad
- Attachment of the WPS on the wall
- Appearance of the WPS cover
- Holding of the cover and the hinges of the WPS
- Cleaning the complete configuration
- Check the label bearing the CE marking

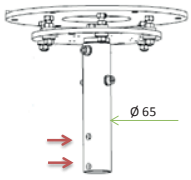
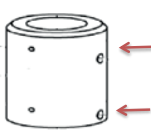
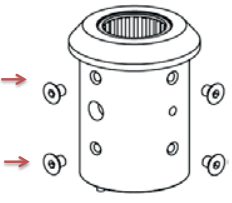
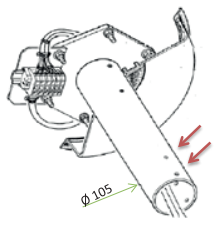
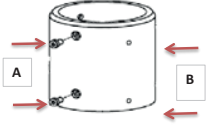
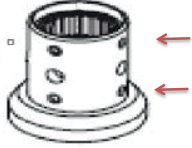
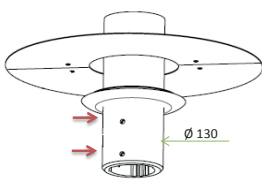
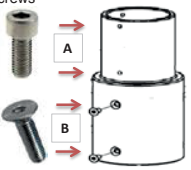
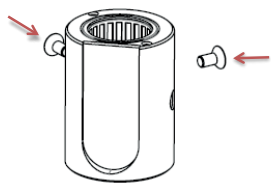
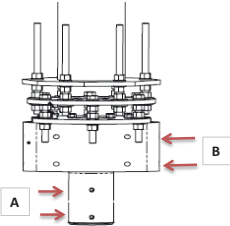
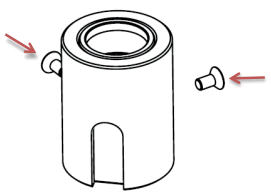
***We do not recommend the use of alcoholized solution***

# SAFETY PARTS TO BE REPLACED EVERY 6 YEARS

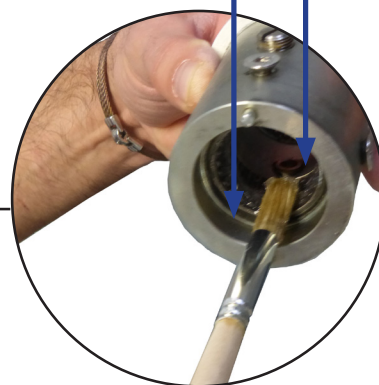
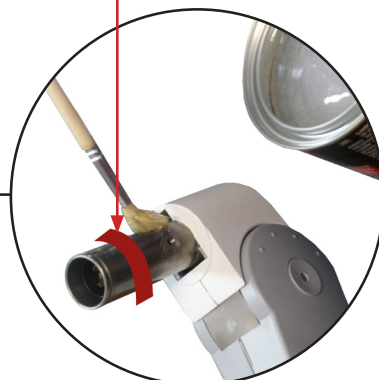
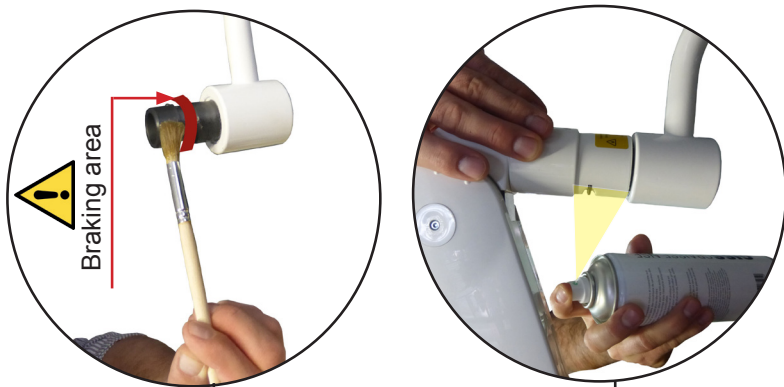
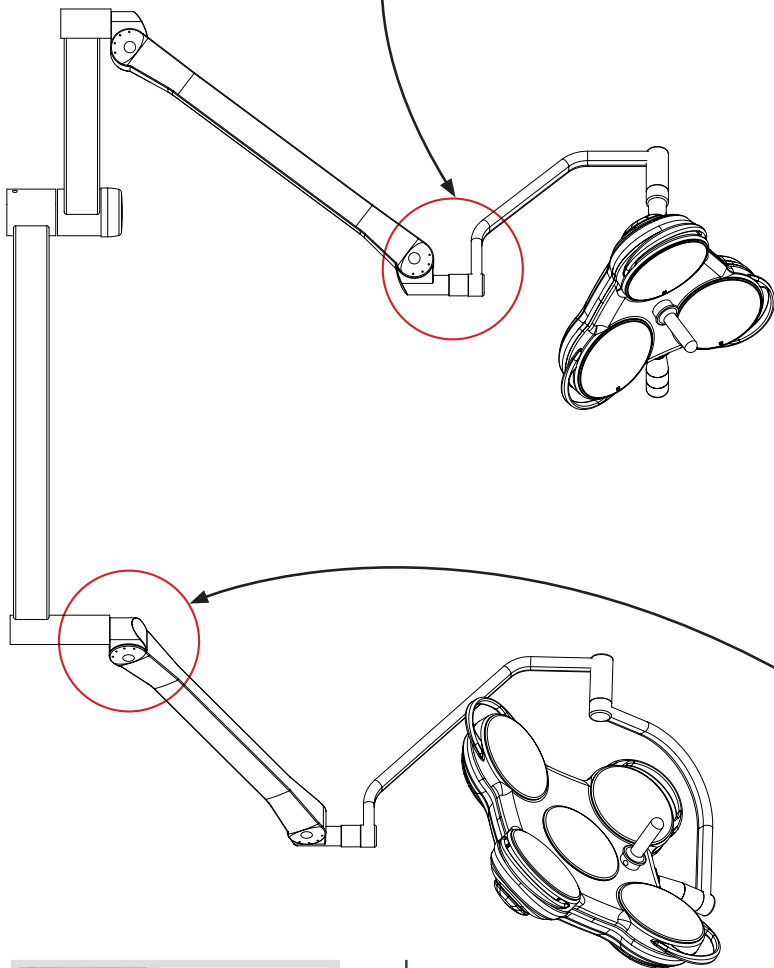
## Safety screws:

Maquet recommends replacing the safety screws every six years as a precaution.

TUBES	ADAPTERS	BUSHINGS
<p><b>TUB 60</b></p> <p>SUSP. BLUE 30-80 SUSP. LCA40-50-100 SUSP. VOLISTA ACCESS</p>  <p>Ø 65</p> <p>P/N : ARD36883555 Kit of four pre-glued M6×16 screws</p> <p>⚠ Fastening torque: 17 Nm</p>	<p>ADPT 120 TO 60 ARD515088999</p>  <p>P/N : ARD315027555 Kit of six pre-glued M6×20 low-profile socket head cap screws</p> <p>⚠ Fastening torque: 14 Nm</p>	<p>SA suspension arm cap</p>  <p>P/N : ARD368325556 Kit of four M6×8 socket head countersunk screws</p> <p>⚠ Fastening torque: 7 Nm</p>
<p><b>ECL</b></p> <p>SUSP. PRC SUSP. PRX (Avant Sept. 2002)</p>  <p>Ø 105</p> <p>P/N : ARD367014555 Kit of six pre-glued M6×12 socket head cap screws</p> <p>⚠ Fastening torque: 14 Nm</p>	<p>Adapter kit ADAPT KIT HLX2 ARD567914999</p>  <p>A = P/N : ARD315027555 Kit of six pre-glued M6×20 low-profile socket head cap screws</p> <p>B = P/N : ARD367014555 Kit of six pre-glued M6×12 socket head cap screws</p> <p>⚠ Fastening torque: 14 Nm</p> <p>Not sold in the US</p>	<p>SATELITE-OSP suspension arm cap</p>  <p>P/N : ARD315028555 SAT-Kit of four screws for OSP end cap</p> <p>⚠ Fastening torque: 7 Nm</p>
<p><b>TUB 130</b></p> <p>SA suspension PRX suspension (after September 2002)</p>  <p>Ø 130</p> <p>P/N : ARD367270555 Kit of six pre-glued M6×20 (class 10.9) socket head cap screws</p> <p>⚠ Fastening torque: 14 Nm</p>	<p>SA suspension adapter on HANAU CITY tube ARD397808555</p> <p>P/N : ARD397808555 Kit of 12 pre-glued socket head cap screws and socket head countersunk screws</p>  <p>⚠ A : Fastening torque: 14 Nm ⚠ B : Fastening torque: 17 Nm</p> <p>Not sold in the US</p>	<p>ACCESS suspension arm end cap</p>  <p>P/N : ARD368840555 Kit of four pre-glued M6×12 socket head countersunk screws</p> <p>⚠ Fastening torque: 17 Nm</p>
<p><b>SATELITE</b></p> <p>Shaft 1: SA suspension Shaft 2: PRX suspension (after September 2002) Shaft 2/3: ARM SAT</p>  <p>A = P/N : ARD315027555 Kit of six pre-glued M6×20 (class 10.9) low-profile socket head cap screws</p> <p>B = P/N : ARD315026555 Kit of six pre-glued M6×25 socket head cap screws</p> <p>⚠ Fastening torque: 14 Nm</p>	<p>LCA DF suspension arm end cap</p>  <p>P/N : ARD368840555 Kit of four pre-glued M6×12 socket head countersunk screws</p> <p>⚠ Fastening torque: 17 Nm</p>	

**CLEAN THE SHAFTS AND JOINTS USING A CLEAN CLOTH BEFORE APPLYING GREASE**

**GREASES USED**

**REMOVE EXCESS GREASE USING A CLEAN CLOTH  
AVOID GREASING THE BRAKING AREAS**

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