



# PiCCO Module

## Short Setup Guide

This document is intended to provide information to an international audience outside of the US.

GETINGE 




## Step 1a: Install PiCCO Module

- Turn the interface plug protection on the rear of the PulsioFlex device away from the plug
- Gently move the module in a horizontal direction until it locks into place
- To unlock and detach the PiCCO module from the device, push the release button on the top of the module and gently remove the module in a horizontal direction from the monitor



## Step 1b: Patient and module wiring

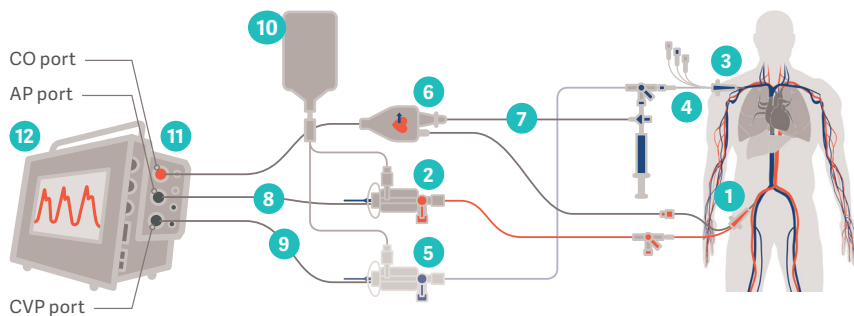

### Product description

### REF


### Getinge order #

	Product description	REF	Getinge order #
1	Arterial thermodilution catheter: PiCCO Catheter	e.g. PV2015L20-A	6885049
2	Arterial pressure transducer: PiCCO Monitoring Kit	e.g. PV8215	6882817
3	Standard CVC	-	-
4	Injectate Temperature Sensor Housing	PV4046	6882773
5	Central venous pressure transducer: PiCCO Monitoring Kit	optional PV8215-2	6882818
6	Arterial Connection Cable	PC80150	6882742
7	Injectate Sensor Cable	PC80109	6882741
8	Pressure Connection Cable	PMK-206	6882743
9	Pressure Connection Cable	optional PMK-206	6882743
10	Flush Bag	-	-
11	PiCCO Module	PC4510	6882748
12	PulsioFlex Monitor	PC4000	6882747




## Step 2: Patient admission

- Turn device on
- Patient screen opens automatically but can also be chosen via main menu
- Enter patient data

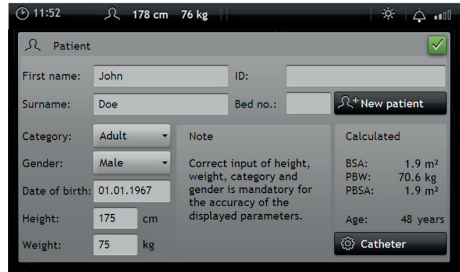

## Step 3: Zero adjustment

- Go to the main menu and open the Zero AP/CVP window
- Open 3 way stopcock to the atmosphere
- Perform zeroing on the PulsioFlex and the bedside monitor
- Open 3 way stopcock to the patient

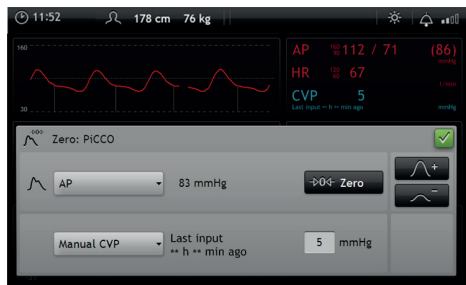

## Step 4: Thermodilution procedure


- Press on the CI field in the main screen or the PiCCO button in the main menu to open the TD (thermodilution) measurement screen
- Complete the TD adjustment






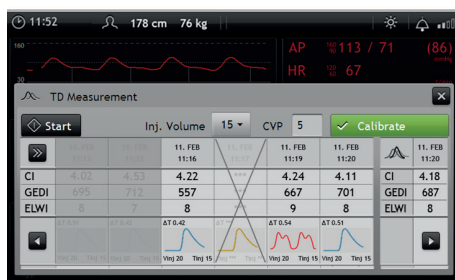
## Step 5: Start thermodilution

- Check injectate volume
- Press start
- When the message "Inject 15 ml" appears, inject the prepared injectate through the Injectate Temperature Sensor Housing

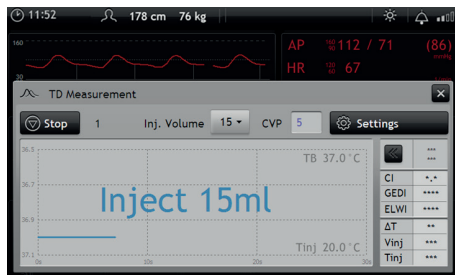

## Step 6: Calibrate PiCCO measurement

- The button  opens the TD results screen (opens automatically when auto TD option used)
- Check measurements
- Measurements excluded from the average are crossed out
- Confirm measurement with „Calibrate“



# GETINGE

This information is intended for an international audience outside the US and does not replace any individual therapeutic decision of the treating physician. Indications, contradictions, warnings and instructions for use are listed in the separate instructions for use. Products may be pending regulatory approvals to be marketed in your country.

PULSION Medical Systems SE · Hans-Riedl-Str. 17 · 85622 Feldkirchen · Germany · +49 89 45 99 14-0  
zentrale.pulsion@getinge.com

[www.getinge.com](http://www.getinge.com)