

XEN

Chemistries
for the ultimate
peace of mind

Xcelerate



Ophthalmic Instruments



Orthopedics



Robotic Instruments



Flexible Scopes



Rigid Scopes



Pretreatment

Instructions for Use

XEN Xcelerate is a ready-to-use, pH-neutral, pretreatment foam formulated for point-of-use treatment of medical instruments and devices, including ophthalmic equipment, prior to transport. Xcelerate penetrates and loosens surgical soils, prevents them from drying and enhances the cleaning process.

The water-based foam spray breaks down quickly for better visibility of sharps.



Order no.	Description
6036002401	Xcelerate 1 x 16 fl. oz.

Directions for Use

1. Place instruments into transport container. Ensure box locks and hinges are open and instruments are separated and disassembled per manufacturer's IFU.
2. Shake bottle before use.
3. Turn the nozzle of the spray head to the ON position.
4. Immediately after use and prior to transport, spray generously on soiled instruments, ensuring complete coverage. If soil has already dried, leave Xcelerate on instruments for a minimum of 1 hour.
5. Prior to the next reprocessing step, rinse instruments thoroughly to remove any residual soils and product.

Caution

- Do not mix with other chemicals.
- Thoroughly rinse instruments prior to subsequent reprocessing step.
- Wear correct PPE and follow applicable guidelines. Additional information on SDS.
- Check material compatibility before use.

Dispose of contents and container in accordance with local, regional, national, and international regulations.

Contains xanthan gum. For more ingredient information, visit www.getinge.com

Storage Information

Store in a cool, dry place protected from frost and away from acids and strong oxidizing agents. Store upright in original containers. Optimum storage temperature 41°F–77°F/5°C–25°C.

	Aluminum	Anodized Aluminum	Titanium	Copper	Brass	Gold	Nickel	Cobalt Chrome	Matte chrome plated	Stainless steel	Glass	Ceramics	Rubber	Plastic	Teflon™/ PTFE	Polycarbonate
Xcelerate	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•

- = The product is suitable for the material
- = Pay special attention to the guidelines from the instruments/device manufacturer

Teflon™ is a trademark of Chemours.

Getinge is a registered trademark of Getinge AB or its subsidiaries or affiliates in the United States or other countries. Copyright 2024 Getinge AB or its subsidiaries or affiliates. • All rights reserved.

Manufacturer: Quadralene Ltd, Unit 3, St. Modwen Park, Andresseway Way, Derby, DE21 6YH, United Kingdom.

Sales Office, US: 1 Geoffrey Way, Wayne, NJ 07470, USA

Sales Office, Canada: 90 Matheson Blvd West, Suite 300, Mississauga, Ontario L5R 3R3, Canada

www.getinge.com

